



Dental Materials, Inc.

RE-GEN Cleanser

Description:

RE-GEN Cleanser is an aqueous cleanser that has been designed to remove contaminants from the surface of a tooth prior to bonding. These contaminants include such items as: tooth debris, temporary cement, bacteria, blood, saliva, hand piece oil, imaging powder, etc.

Along with cleansing the surface, RE-GEN Cleanser will etch tooth surfaces to increase bond strengths of self-etch materials. RE-GEN Cleanser properly prepares the enamel surface, resulting in improved long-term results. While cleaning and etching, RE-GEN Cleanser disinfects the surfaces being treated.

RE-GEN Cleanser is completely rinsed from the surface after use and thus is compatible with all other dental materials.

Principal uses of RE-GEN Cleanser:

- Cleansing dentin and enamel surfaces
- Removing imaging powder from all surfaces
- Disinfecting tooth surfaces
- Etching tooth structure for improved adhesion and long-term results

Instructions for use:

Syringe:

1. With the enclosed brush tips, apply RE-GEN Cleanser to the surface to be cleansed. Surface should be completely saturated with the RE-GEN Cleanser.
2. Using the delivery brush tip, agitate the RE-GEN Cleanser on the surface for 10 seconds
3. Rinse thoroughly

Storage:

1. Protect from sunlight and excessive temperatures (<30°C/85°F)
2. Store at room temperature. Do not freeze.
3. Refrigerate materials that are not to be used for more than 7 days



Dental Materials, Inc.

RE-GEN Cleanser



WARNING

Causes serious eye irritation.

Wear eye/face protection. Prolonged or repeated skin contact may cause severe irritation.

FIRST AID:

IF IN EYES: Rinse cautiously with water for several minutes, also under eye lids. Remove contact lenses, if present and easy to do so. **IF INHALED:**

Move to fresh air and keep at a rest position comfortable for breathing. **IF**

INGESTED: Do not induce vomiting. Rinse mouth, drink plenty of water.

Call a physician or Poison Control Center immediately.



Inter-Med, Inc.

2200 South St.
Racine, WI 53404
Tel: +1-262-636-9755



Emergo Europe

Prinsessegracht 20
2514 AP The Hague
The Netherlands

91035-I-AP-ENG (1)