

**Description:**

Stat-Pak™ Knitted Retraction Cord is used to retract sulcular tissue, protect tissues and arrest sulcular fluid seepage. Easy to control and place, knitted Stat-Pak™ Dental Retraction Cord carries more hemostatic agent to the tooth preparation and remains in-situ, even when touched by a bur or diamond.

Indication for Use:

- Stat-Pak™ Knitted Retraction Cord is used to retract sulcular tissue, protect tissues and arrest sulcular fluid seepage.

Precautions:

1. **DO NOT** reuse.

Instructions for Use:

1. Prepare crown as usual.
2. Select a thin packer to place the cord.
The interlocking loops prevent instrument from penetrating cord.
3. Pack cord into the sulcus, distending into the tissue.
Limit to one quadrant of the mouth at a time.
4. Follow packing, the loops maintain a gentle, continuous, outward force against the gingival cuff. Leave 3-minutes.
5. Remove cord and spray with air/water rinse.
6. If bleeding is still persistent, repeat steps until hemostasis is achieved.
7. Rinse, dry, make impression

