

Stat-Pak™

KNITTED RETRACTION CORD

REF	302611	Size 000 Black
REF	302612	Size 00 Brown/Yellow
REF	302613	Size 0 Purple
REF	302614	Size 1 Blue
REF	302615	Size 2 Green

Description:

Stat-Pak™ Knitted Retraction Cord is used to retract sulcular tissue, protect tissues and arrest sulcular fluid seepage. Easy to control and place, Stat-Pak™ Knitted Retraction Cord carries more hemostatic agent to the tooth preparation and remains in-situ, even when touched by a bur or diamond.

Indication For Use:

Stat-Pak™ Knitted Retraction Cord is used to retract sulcular tissue, protect tissues and arrest sulcular fluid seepage.

Precautions:

DO NOT reuse. *Caution:* Sharp cutter under cap. Close cap before disposal.

Instructions For Use:

1. Prepare crown as usual.
2. Select a thin packer to place the cord.
The interlocking loops prevent instrument from penetrating cord.
3. Pack cord into the sulcus, distending into the tissue.
Limit to one quadrant of the mouth at a time.
4. Follow packing, the loops maintain a gentle, continuous, outward force against the gingival cuff.
Leave for 3 minutes.
5. Remove cord and spray with air/water rinse.
6. If bleeding is still persistent, repeat steps until hemostasis is achieved.
7. Rinse, dry, make impression.



Do Not
Use If
Package Is
Damaged



Do Not
Reuse



Temperature
Range For
Storage



Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel.: +1-262-636-9755
www.vistaapex.com

Made in South Korea
302600-I-V-ENG (3)

