



# Stat-Pak™+E

---

## Impregnated Retraction Cord

### INSTRUCTIONS FOR USE

**Stat-Pak™+E** is braided cotton cord saturated with a solution of racemic epinephrine hydrochloride and dried by a technique which retains the activity of the medication. Normal tissue moisture reactivates the medication, which promptly retracts gingival tissue in a manner that causes no tissue damage, yet fully exposes the gingival margin. All **Stat-Pak™+E** retraction cords are braided to prevent fraying, unraveling and penetration of the instrument through the cord while packing. These qualities contribute to easy and proper packing.

**Contents:** Each bottle contains 72 inches braided cotton cord.

**Average concentration:** Epinephrine in **Stat-Pak™+E**  
size 00 = 0.30 mg/in; size 0 = 0.50 mg/in; size 1 = 0.85 mg/in.

**Indications for Use:** Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation.

### CONTRAINDICATIONS:

**Stat-Pak™+E** is contraindicated for persons who are epinephrine sensitive, or for persons suffering from cardiovascular disease, diabetes, or hyperthyroidism. Caution should also be taken for elderly, pregnant, and pediatric patients.

**Warnings:** Contains Sodium Bisulfite, a sulfite that may cause allergic-type reactions, including anaphylactic symptoms, and life-threatening or less severe asthmatic episodes, in certain susceptible persons. The overall prevalence of sulfite sensitivity in the general population is unknown, but probably low. Sulfite sensitivity is seen more in asthmatic than non-asthmatic persons.

**Precautions: Read and follow all instructions.**

For professional in-office use only • For topical intraoral use only • Do not ingest • Keep out of reach of children  
Do not use on patients with Epinephrine allergy or sensitivity.

**AMOUNT TO USE:**


Use as much cord of the appropriate sizes as is necessary to go all the way around the neck of each tooth to be retracted, with a slight overlap. Two loops may be packed around each tooth (see below), if sulcus space allows. Retraction with an epinephrine cord should be limited to the equivalent of one normal quadrant of the mouth per appointment.

**DIRECTIONS FOR USE:**

For most effective use of **Stat-Pak™+E**, loop a strand of smaller cord around tooth and over the crevice.

- Trim ends for slight overlap; pack the loop firmly into sulcus.
- Apply a second loop of larger cord over the smaller cord in the same manner.
- When ready to apply the impression material, remove larger cord loop only leaving the smaller cord in place below the gingival margin until impression procedure is complete.
- Once impression procedure has been completed the smaller cord is removed also. Alternately, a single loop of cord will suffice.

**Stat-Pak™+E** completely retracts tissue within five to eight minutes and should then be removed from the mouth. Dispensed portions of cord are intended for single use only and should be disposed of immediately after use. For product MSDS information please contact Vista at 1-877-4184782. Stat-Pak™ +E is free of gluten, soy, milk, egg, peanut, and tree nut products. Stat-Pak +E contains no artificial flavors or colors.

 **Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755  
[www.vistaapex.com](http://www.vistaapex.com)



**Rx ONLY**

302631-I-V-ENG (5)