

Stat-Pak™ +E

BRAIDED RETRACTION CORD WITH EPINEPHRINE

Instructions For Use:

Stat-Pak™ +E is braided cotton cord saturated with a solution of racemic epinephrine hydrochloride and dried by a technique which retains the activity of the medication. Normal tissue moisture reactivates the medication, which promptly retracts gingival tissue in a manner that causes no tissue damage, yet fully exposes the gingival margin. All Stat-Pak™ +E retraction cords are braided to prevent fraying, unraveling and penetration of the instrument through the cord while packing. These qualities contribute to easy and proper packing.

Contents:

Each bottle contains 96 inches (244 cm) braided cotton cord.

Average concentration:

Epinephrine in Stat-Pak™ +E Braided Retraction Cord:

- Size 00 = 0.30 mg/in
- Size 0 = 0.50 mg/in
- Size 1 = 0.85 mg/in
- Size 2 = 1.15 mg/in

Indications for Use:

Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation.

Contraindications:

Stat-Pak™ +E is contraindicated for persons who are epinephrine sensitive, or for persons suffering from cardiovascular disease, diabetes, or hyperthyroidism. Caution should also be taken for elderly, pregnant, and pediatric patients.

Warnings:

Contains Sodium Bisulfite, a sulfite that may cause allergic-type reactions, including anaphylactic symptoms, and life-threatening or less severe asthmatic episodes, in certain susceptible persons. The overall prevalence of sulfite sensitivity in the general population is unknown, but probably low. Sulfite sensitivity is seen more in asthmatic than non-asthmatic persons.

Precautions: Read and follow all instructions.

- For professional in-office use only
- For topical intraoral use only
- Do not ingest
- Keep out of reach of children
- Do not use on patients with Epinephrine allergy or sensitivity

Amount To Use:

Use as much cord of the appropriate sizes as is necessary to go all the way around the neck of each tooth to be retracted, with a slight overlap. Two loops may be packed around each tooth, if sulcus space allows. Retraction with an epinephrine cord should be limited to the equivalent of one normal quadrant of the mouth per appointment.

Directions For Use:

For most effective use of Stat-Pak™ +E, loop a strand of smaller cord around tooth and over the crevice.

- Trim ends for slight overlap; pack the loop firmly into sulcus.
- Apply a second loop of larger cord over the smaller cord in the same manner.
- When ready to apply the impression material, remove larger cord loop only leaving the smaller cord in place below the gingival margin until impression procedure is complete.
- Once impression procedure has been completed the smaller cord is removed also. Alternately, a single loop of cord will suffice.

Stat-Pak™ +E completely retracts tissue within five to eight minutes and should then be removed from the mouth. Dispensed portions of cord are intended for single use only and should be disposed of immediately after use. For product MSDS information please contact VistaApex at 1-877-418-4782. Stat-Pak™ +E is free of gluten, soy, milk, egg, peanut, and tree nut products. Stat-Pak™ +E contains no artificial flavors or colors.



Do Not
Use If
Package Is
Damaged



Temperature
Range For
Storage



Do Not
Reuse



Caution

Rx ONLY

Prescription
Required



Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel.: +1-262-636-9755
www.vistaapex.com

Made in USA
302631-I-V-ENG (6)

