VISTA | APEX

Stropko Irrigator™

Irrigation and Aeration
Luer-Lock Tip Adapter

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EN This instrument can be used with most air / water syringes using the 3-way dental syringe tip.

Indications for Use:

The product mates an air/water syringe to luer lock style needle tips for precise control of air and water during dental procedures.

Indications for Use:

- The air pressure to the syringe must be regulated down to between 4-7 psi. (normal is 30-50 psi)
- When using for air-drying endodontic canals, reduce the air pressure to 1-3 psi and use only notched or side-vent needles
- If the syringe has a "Quick Connector" remove the standard tip and replace it with the Stropko Irrigator™. Insert Stropko Irrigator™ snugly into the air/water syringe. (If using press ring style air/water handpiece, ensure that Stropko Irrigator™ is locked in.)
- Use any luer-lock needle or tip. The desired tip must be completely screwed into the luer-lock hub on the Stropko Irrigator™ to prevent dislodgement during use
- The luer tips used cannot have any extensions back into the instrument beyond the hub. Do NOT use sharp pointed injection-type needles or needle tips that are smaller than 27 gauge.
- Autoclave after each use at 132°C for 4 minutes.

Note - According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

Note: Older 3-way syringes must have a small pin inserted below and passing behind the air and water buttons to prevent them from "popping out". If the pin is not present, a newer version of the 3-way syringe must be acquired

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