



VISTA APEX

Vista-BLUE™

REF 317007 - Unit Dose, 20pk



**Rx ONLY**

*Harmful if swallowed. Avoid contact with skin and eyes.*

### Description:

Vista-BLUE™ is a water soluble solution which is used for the identification of cracks, calcified canals, root canal orifices and carious dentin.

### Method:

1. Remove Vista-BLUE™ unit dose applicator and reservoir from packaging.
2. Gently twist and push grey micro applicator brush into reservoir.
3. Remove Micro-Applicator brush.
4. Apply Vista-BLUE™ onto the tooth surface to be checked.
5. Wait 5 – 10 seconds and rinse with water.
6. Identify the crack in the tooth.
7. Re-dip micro brush as needed and repeat steps 4 – 6.
8. Dispose of unit dose applicator and reservoir after every patient.

### Storage:

1. May stain clothing and skin.
2. Avoid contact with skin and eyes.
3. In the event of eye contact, flush with copious amounts of water and seek immediate medical attention.
4. If contact with clothing occurs, stain may be removed with mild detergent and warm water.
5. Keep refrigerated. Protect from freezing.

**NOTE:** See [vistaapex.com](http://vistaapex.com) for SDS.



**Inter-Med, Inc.**

2200 South St., Racine, WI 53404, USA

Tel: +1-262-636-9755

[www.vistaapex.com](http://www.vistaapex.com)

**Made in USA**  
317007-I-V-ENG (5)