




VISTA | APEX

PATENTED

SmearOFF™

-2 IN 1-

*EDTA-based formula
enhanced with Chlorhexidine*

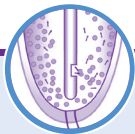

REF 317112

Use after sodium hypochlorite for one-step smear layer removal and long-lasting cleansing.

4 oz (120mL) VOLUME

SmearOFF™ Indications

SmearOFF™ 2-in-1 solution is used after endodontic instrumentation for the removal of the smear layer and dentin mud, and final rinse cleansing. SmearOFF™ is an EDTA based formula enhanced with chlorhexidine for added cleansing properties.

Instructions for Use:

1. Irrigate with Chlor-XTRA™ (enhanced 6% NaOCl) as needed for debridement during instrumentation, prior to use of SmearOFF™.
2. Fill a clean syringe with SmearOFF™ solution
3. Attach an irrigation needle tip securely to syringe (e.g. 30ga Vista-Probe™ or Appli-Vac™ irrigating tip)
4. Irrigate for 60-90 seconds
5. Irrigate again with Chlor-XTRA™ for further debridement.
6. Finish with a final rinse of SmearOFF™ for optimal canal cleanliness
7. Evacuate the canal with a micro suction tip (e.g. Elasti-Vac™ Tip)
8. Blot out the canal using 1-2 paper points
9. Seal and obturate as normal

! WARNING

- May be harmful if swallowed. Causes mild skin irritation.
- Causes eye irritation. Wash hands thoroughly after handling.
- Wear protective gloves/eye glasses. Follow all government regulations for waste disposal. Prevent release to the environment if possible. Do not flush waste into sewer or waterways. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing.
- IF ON SKIN:** Wash with plenty of soap and water.

Rx ONLY



Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755
www.vistaapex.com

Emergo Europe
Prinseessegracht 20
2514 AP The Hague
The Netherlands

317112-V-ENG (4)
Made in USA