

VISTA | APEX



dripless syringe

Patent Pending

INSTRUCTIONS FOR USE

**FOR A DRIPLESS SYRINGE TO FUNCTION PROPERLY,
ALL AIR BUBBLES MUST BE REMOVED FROM THE SYRINGE.**

1. Draw syringe plunger back to fill with desired irrigating solution.
2. Attach irrigating tip to the syringe.
3. Hold the syringe with the needle pointing straight up.
4. Gently tap the barrel of the syringe so air bubble float to the top.
5. Still holding the syringe upright, slowly push the plunger until you push all the air out of the syringe.



REF 318060, 318012

INTENDED FOR SHORT TERM STORAGE OF NaOCl (2 WEEKS OR LESS)

Designed by and exclusively
manufactured for:

Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
www.vistaapex.com

Rx ONLY



**WUXI YUSHOU MEDICAL
APPLIANCES CO., LTD.**
No. 115 Nongxinhe Rd.
Xishan District
214191 Wuxi/Jiangsu, China
Tel: 0086-510-83777555



**Shanghai International Holding
Corp. GmbH (Europe)**
Eiffestrasse 80, 20537
Hamburg Germany
Tel: +49-40-2513175
Fax: +49-40-255726

