



# dripless syringe

Patent Pending

## INSTRUCTIONS FOR USE

**FOR A DRIPLESS SYRINGE TO FUNCTION PROPERLY,  
ALL AIR BUBBLES MUST BE REMOVED FROM THE SYRINGE.**

1. Draw syringe plunger back to fill with desired irrigating solution.
2. Attach irrigating tip to the syringe.
3. Hold the syringe with the needle pointing straight up.
4. Gently tap the barrel of the syringe so air bubbles float to the top.
5. Still holding the syringe upright, slowly push the plunger until you push all the air out of the syringe.



**INTENDED FOR  
SHORT TERM STORAGE  
OF NaOCl (2 WEEKS OR LESS)**



Inter-Med, Inc.  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755  
www.vistadental.com

**Made in USA**



**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
Tel: +31(0)70 345 8570  
Fax: +31(0)70 346 7299



**Rx ONLY**

318000-I-V-ENG (5)