VISTA | APEX

**NiTi**  
**SuperFlex™ Tips**  
**(Patented)**

**REF** 405107, 405106  
405108

**Rx Only**

**Made in China**

**CE**  
0459

**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755  
[www.vistaapex.com](http://www.vistaapex.com)

405107-I-V-MUL-(1)

**REF**Catalogue  
Number

EN	Catalogue number
BG	Каталожен номер
CZ	Katalogové číslo
DA	Katalognummer
ET	Kataloogi number
FR	Référence catalogue
DE	Bestellnummer
EL	Αριθμός καταλόγου
HU	Katalógusszám
IT	Numero di catalogo
LV	Kataloga numurs
LT	Katalogo numeris
PT	Número de catálogo
RO	Număr de catalog
SK	Katalógové číslo
ES	Número de catálogo
SV	Katalognummer

Do not use  
if package is  
damaged

EN	Do not use if package is damaged
BG	Не използвайте, ако опаковката е повредена
CZ	Nepoužívejte, pokud je obal poškozen
DA	Må ikke anvendes, hvis pakningen er beskadiget
ET	Ärge kasutage, kui pakend on kahjustatud
FR	Ne pas utiliser si l'emballage est endommagé
DE	Nicht verwenden, wenn die Verpackung beschädigt ist
EL	Μην το χρησιμοποιείτε εάν η συσκευασία είναι κατεστραμμένη
HU	Ne használja, ha a csomagolás sérült
IT	Non utilizzare se la confezione è danneggiata
LV	Nelietot, ja iepakojums ir bojāts
LT	Nenaudkite, jei pakuotė pažeista
PT	Não use se o pacote estiver danificado
RO	Nu utilizați dacă ambalajul este deteriorat
SK	Nepoužívejte, ak je obal poškodený
ES	No utilice el producto si el envoltorio está dañado
SV	Får inte användas om förpackningen är skadad

Follow  
Instructions  
For Use

EN	Follow instructions for use
BG	Следвайте инструкциите за употреба
CZ	Postupujte podle návodu k použití
DA	Følg brugsanvisningen
ET	Järgige kasutusjuhiseid
FR	Suivez les instructions d'utilisation
DE	Gebrauchsanweisung befolgen
EL	Ακολουθήστε τις οδηγίες χρήσης
HU	Kövesse a használati utasításokat
IT	Attenersi alle istruzioni per l'uso
LV	Izpildiet lietošanas instrukcijas
LT	Vadovaukitės naudojimo instrukcijomis
PT	Siga as instruções de uso
RO	Urmați instrucțiunile de utilizare
SK	Postupujte podľa návodu na použitie
ES	Siga las instrucciones de uso
SV	Följ bruksanvisningen

132 °C

Autoclavable  
Up To The  
Temperature  
Specified

EN	Autoclavable up to the temperature specified
BG	Автоклавируема до зададената температура
CZ	Autoklávovatelný až do stanovené teploty
DA	Autoklaverbar op til den angivne temperatur
ET	Autoslaveeritav kuni määratud temperatuurini
FR	Autoclavable jusqu'à la température spécifiée
DE	Autoklavierbar bis zur angegebenen Temperatur
EL	Αυτοκλείστο μέχρι τη θερμοκρασία που καθορίζεται
HU	Autoklávitható a megadott hőmérsékletig
IT	Autoclavabile fino alla temperatura specificata
LV	Autoklāvējams līdz norādītajai temperatūrai
LT	Autoklavuojamas iki nurodytos temperatūros
PT	Autoclavável até a temperatura especificada
RO	Autoclavabil până la temperatura specificată
SK	Autoklávovatelné až do špecifikovanej teploty
ES	Esterilizable en autoclave hasta la temperatura especificada
SV	Autoklaverbar upp till den angivna temperaturen

Manufacturer

EN	Manufacturer
BG	Производител
CZ	Výrobce
DA	Producent
ET	Tootja
FR	Fabricant
DE	Hersteller
EL	Κατασκευαστής
HU	Gyártó
IT	Produttore
LV	Ražotājs
LT	Gamintojas
PT	Fabricante
RO	Producător
SK	Výrobca
ES	Fabricante
SV	Tillverkare

Authorized Representative in the European Community

EN	Authorized Representative in the European Community
BG	Упълномощен представител в Европейската общност
CZ	Zplnomocněný zástupce v Evropském společenství
DA	Autoriseret repræsentant i EU
ET	Volitatud esindaja Euroopa Ühenduses
FR	Représentant agréé pour la Communauté européenne
DE	Autorisierte Vertretung in der Europäischen Gemeinschaft
EL	Εξουσιοδοτημένος Αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα
HU	Az Európai Közösség meghatalmazott képviselője
IT	Rappresentante autorizzato nella Comunità Europea
LV	Pilnvarotais pārstāvis Eiropas Kopienā
LT	Igaliotasis atstovas Europos bendrijoje
PT	Representante Autorizado na Comunidade Europeia
RO	Reprezentant autorizat în Comunitatea Europeană
SK	Zplnomocnený zástupca v Európskom spoločenstve
ES	Representante autorizado en la Comunidad Europea
SV	Auktoriserad representant i EU

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**EN****Indications for use:**

Applying endodontic irrigation solutions to the root canal.

**Instructions for use:**

- 1) Rotate needle tip hub firmly on prefilled luer lock syringe
- 2) Gently depress plunger to deliver solution
- 3) Remove tip from syringe

**To clean tip:****Do not allow tips to exceed 125°C.**

1. Soak in a mild, pH-neutral detergent until cleaning can be performed.
2. Rinse tips under a hard stream of water to remove debris on the outside surface of the tip.
3. Flush tip with a syringe filled with distilled water or alcohol to clear the tip.
4. Clean tip in an ultrasonic unit according to unit manufacturer's instructions using a low-foaming pH-neutral detergent. Use distilled water if possible. Do not allow tips to contact any other metals. Thoroughly rinse tips according to instructions supplied by detergent manufacturer.
5. Wrap tips for sterilization with wraps washed in pH-neutral detergents or disinfectants. Do not use commercial wraps with sterilization indicators that leave stains. Do not mix tips with other metals during autoclaving, or allow them to directly touch metal trays. Steam autoclave at 121°C/250°F, 15-20 psig for 30 minutes.

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