VISTA APEX

NiTi
SuperFlex™
Irrigating Tips

Patented

REF 405106,
405107, 405108

Rx ONLY

Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755
www.vistaapex.com

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

0459

Made in China

405107-I-V-MUL (4)

REF
Catalogue
Number

EN	Catalogue number
BG	Каталожен номер
CZ	Katalógové číslo
DA	Katalognummer
ET	Kataloogi number
FR	Référence catalogue
DE	Bestellnummer
EL	Αριθμός καταλόγου
HU	Katalógusszám
IT	Numero di catalogo
LV	Kataloga numurs
LT	Katalogo numeris
PT	Número de catálogo
RO	Număr de catalog
SK	Katalógové číslo
ES	Número de catálogo
SV	Katalognummer

Do not use
if package is
damaged

EN	Do not use if package is damaged
BG	Не използвайте, ако опаковката е повредена
CZ	Nepoužívejte, pokud je obal poškozen
DA	Må ikke anvendes, hvis pakningen er beskadiget
ET	Ärge kasutage, kui pakend on kahjustatud
FR	Ne pas utiliser si l'emballage est endommagé
DE	Nicht verwenden, wenn die Verpackung beschädigt ist
EL	Μην το χρησιμοποιείτε εάν η συσκευασία είναι κατεστραμμένη
HU	Ne használja, ha a csomagolás sérült
IT	Non utilizzare se la confezione è danneggiata
LV	Nelietot, ja iepakojums ir bojāts
LT	Nenaudkite, jei pakuotė pažeista
PT	Não use se o pacote estiver danificado
RO	Nu utilizați dacă ambalajul este deteriorat
SK	Nepoužívejte, ak je obal poškodený
ES	No utilice el producto si el envoltorio está dañado
SV	Får inte användas om förpackningen är skadad

Follow
Instructions
For Use

EN	Follow instructions for use
BG	Следвайте инструкциите за употреба
CZ	Postupujte podle návodu k použití
DA	Følg brugsanvisningen
ET	Järgige kasutusjuhiseid
FR	Suivez les instructions d'utilisation
DE	Gebrauchsanweisung befolgen
EL	Ακολουθήστε τις οδηγίες χρήσης
HU	Kövesse a használati utasításokat
IT	Attenersi alle istruzioni per l'uso
LV	Izpildiet lietošanas instrukcijas
LT	Vadovaukitės naudojimo instrukcijomis
PT	Siga as instruções de uso
RO	Urmați instrucțiunile de utilizare
SK	Postupujte podľa návodu na použitie
ES	Siga las instrucciones de uso
SV	Följ bruksanvisningen

Autoclavable
Up To The
Temperature
Specified

EN	Autoclavable up to the temperature specified
BG	Автокламируема до зададената температура
CZ	Autoklávovatelný až do stanovené teploty
DA	Autoklaverbar op til den angivne temperatur
ET	Autoslaveeritav kuni määratud temperatuurini
FR	Autoclavable jusqu'à la température spécifiée
DE	Autoklavierbar bis zur angegebenen Temperatur
EL	Αυτοκλείστο μέχρι τη θερμοκρασία που καθορίζεται
HU	Autoklávitható a megadott hőmérsékletig
IT	Autoclavabile fino alla temperatura specificata
LV	Autoklāvējams līdz norādītajai temperatūrai
LT	Autoklavuojamas iki nurodytos temperatūros
PT	Autoclavável até a temperatura especificada
RO	Autoclavabil până la temperatura specificată
SK	Autoklávovatelné až do špecifikovanej teploty
ES	Esterilizable en autoclave hasta la temperatura especificada
SV	Autoklaverbar upp till den angivna temperaturen

Manufacturer

EN	Manufacturer
BG	Производител
CZ	Výrobce
DA	Producent
ET	Tootja
FR	Fabricant
DE	Hersteller
EL	Κατασκευαστής
HU	Gyártó
IT	Produttore
LV	Ražotājs
LT	Gamintojas
PT	Fabricante
RO	Producător
SK	Výrobca
ES	Fabricante
SV	Tillverkare

Authorized Representative in the European Community

EN	Authorized Representative in the European Community
BG	Упълномощен представител в Европейската общност
CZ	Zplnomocněný zástupce v Evropském společenství
DA	Autoriseret repræsentant i EU
ET	Volitatud esindaja Euroopa Ühenduses
FR	Représentant agréé pour la Communauté européenne
DE	Autorisierte Vertretung in der Europäischen Gemeinschaft
EL	Εξουσιοδοτημένος Αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα
HU	Az Európai Közösség meghatalmazott képviselője
IT	Rappresentante autorizzato nella Comunità Europea
LV	Pilnvarotais pārstāvis Eiropas Kopienā
LT	Igaliotasis atstovas Europos bendrijoje
PT	Representante Autorizado na Comunidade Europeia
RO	Reprezentant autorizat în Comunitatea Europeană
SK	Zplnomocnený zástupca v Európskom spoločenstve
ES	Representante autorizado en la Comunidad Europea
SV	Auktoriserad representant i EU

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EN**Indications for Use:**

Applying endodontic irrigation solutions to the root canal.

Instructions for Use:

1. Rotate needle tip hub firmly on prefilled luer-lock syringe.
2. Gently depress plunger to deliver solutions.
3. Remove tip from syringe.

Follow detailed reprocessing instructions contained within this IFU for cleaning and sterilizing the device between patients.

Reprocessing Instructions:

Manufacturer:	Inter-Med, Inc.
Device:	NiTi SuperFlex™ Tips
WARNINGS	
<ul style="list-style-type: none"> • Appropriate PPE (gloves, safety glasses, scrubs or lab coat) should be worn when reprocessing medical devices. • Do not disassemble devices. • Do not use abrasive cleaners. • Use neutral pH detergents during cleaning. • Do not use with commercial sterilization wraps or pouches with sterilization indicators that leave a stain. • For autoclavable products, do not sterilize with other products. Do not allow products to touch one another during autoclaving. • Reprocess products as soon as possible after use. • It is the user's responsibility to ensure that all reprocessing equipment used is properly validated, maintained, and calibrated in accordance with equipment specification and requirements. • Reference should be made to the device's instructions for use / user manual for additional information which may be applicable to reprocessing. 	
Limitations on process:	
<ul style="list-style-type: none"> • The products should be thoroughly inspected and tested for functionality after each reprocessing cycle. Devices should be replaced if any visible wear is seen, or if any noticeable decrease in product functionality is observed. Operator use and care will impact how long product will last. Contact Vista Apex with any questions. • Vista Apex has validated the following number of reprocessing cycles for the products: <ul style="list-style-type: none"> - NiTi SuperFlex™ Tips: 80 cycles 	

INSTRUCTIONS**Initial treatment at the point of use:**

- Following clinical use, the product should be immediately rinsed with water and any channels should be flushed. Remove any visible debris or contamination using a lint free cloth.
- If reprocessing cannot be performed immediately, soak product in a mild, pH-neutral detergent until cleaning can be performed and maintain separation of soiled product from non-contaminated devices to avoid cross-contamination.

Preparation before cleaning:

- No disassembly is required for reprocessing. Do not attempt to disassemble product.
- Remove any visible gross debris and wipe using a lint free cloth. For non-electronic devices, water may also be used.

Cleaning & Disinfection: Automated

- It is recommended to use a washer-disinfector meeting the requirements of ISO 15883. Products should be positioned such that they do not make physical contact with other parts in the washer-disinfector. Use a neutral-pH based cleaning agent. There are no known constraints on pressures and temperatures.

Cleaning & Disinfection: Manual

- If cleaning and disinfecting the device manually, thoroughly wipe device using a single-use cloth dampened with a pH-neutral, aldehyde and alkylamine-free, alcohol-based, bactericidal, virucidal, fungicidal disinfectant solution (e.g. Alpet D2 Disinfectant or equivalent) following the manufacturer's instructions for use. Wipe device on all surfaces. Redampen cloth as needed.
- After 1-3 minutes of contact time, remove the disinfectant solution residue with a damp cloth. Use deionized water to dampen cloth.

NOTE: Any lumens, grooves, or fluid paths should be cleaned in an equivalent manner using an appropriately sized non-abrasive cleaning brush following the same steps above.

Drying:

- Dry with gauze or lint-free cloth. Air dry until liquid is no longer present (approximately 30 minutes).

Maintenance, inspection and testing:

- Thoroughly inspect and test the product for functionality after each reprocessing cycle. Confirm product is free of debris and not damaged. Devices should be replaced if any visible wear is seen, or if any noticeable decrease in product functionality is observed.

Packaging:
• No special packaging is required for sterilization.
Sterilization:
• Moist heat sterilize in accordance with ISO 17665-1 at 121°C for at least 30 minutes (15 minute dry time).
Storage:
Store in a clean and dry location. Maintain separation of soiled from non-contaminated devices to avoid cross contamination.
Additional information:
No additional information needed for reprocessing.
Manufacturer contact:
Inter-Med, Inc. 2200 South Street Racine, WI 53404 USA +1-262-636-9755 info@vista-dental.com

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Inter-Med, Inc. 2200 South Street Racine, WI 53404 EE. UU. +1-262-636-9755 info@vista-dental.com