



- Activator tips resonate down entire length of tip
- Depth markers at 16mm, 17mm and 18mm
- Tip frequency: 40-50kHz (40,000-50,000 cycles/sec)


## Installation Instructions:

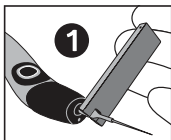

- 1.** Hand thread tip onto handpiece.

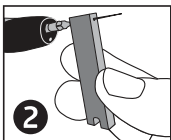

- 2.** Tighten with wrench until tip no longer spins.  
**NOTE:** Do not over-tighten.

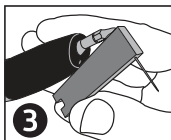

- 3.** Slide tip through wrench hole until it stops.  
**BEND TIP TO DESIRED ANGLE**  
(70° - 90° for best results).

## MAINTENANCE & CARE

- EndoUltra™ tips can be autoclaved prior to use.
- Can withstand autoclave at 132 °C / 0.22 MPa for 3 minutes.



Detailed manual cleaning and sterilization processes are available at: [vista-dental.com](http://vista-dental.com).

**NOTE:** Tips consist of titanium.



**RxOnly**— For dental use only.

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.



**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404  
Tel: +1-262-636-9755



**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands