




VISTA | APEX

## EndoUltra™ Activator Tips


REF 407706

- Activator tips resonate down entire length of tip
- Depth markers at 16mm, 17mm and 18mm
- Tip frequency: 40-50kHz (40,000-50,000 cycles/sec)


### Installation Instructions:

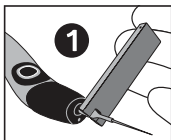

1. Hand thread tip onto handpiece.

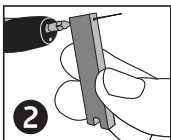

2. Tighten with wrench until tip no longer spins.  
**NOTE:** Do not over-tighten.

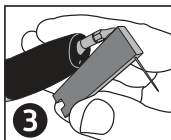

3. Slide tip through wrench hole until it stops.  
**BEND TIP TO DESIRED ANGLE**  
(70° - 90° for best results).

### MAINTENANCE & CARE

- EndoUltra™ tips can be autoclaved prior to use.
- Can withstand autoclave at 132°C / 0.22 MPa for 3 minutes.



Detailed manual cleaning and sterilization processes are available at: [vistaapex.com](http://vistaapex.com).

**NOTE:** Tips consist of titanium.



Inter-Med, Inc.  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755  
[www.vistaapex.com](http://www.vistaapex.com)

**RxOnly** — For dental use only.

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

407703-I-V-ENG (9)