



VISTA | APEX



## Sodium Hypochlorite Pre-filled Syringes

**REF** 502355 - 3% 3mL Pre-filled Syringes

**REF** 502350 - 6% 3mL Pre-filled Syringes

**REF** 502315 - 3% 12mL Pre-filled Syringes

**REF** 502300 - 6% 12mL Pre-filled Syringes

### Description:

Sodium Hypochlorite is an aqueous solution.

It is used for the irrigation and debridement of root canals during and after instrumentation.

### Instructions for Use:

1. Remove syringe cap and attach irrigating tip securely to syringe.
2. Irrigate with Sodium Hypochlorite as needed for debridement during instrumentation.
3. Remove smear layer with appropriate irrigant, such as EDTA solution, SmearOFF™ 2-in-1 or Citric Acid 12.5%.
4. Irrigate again with Sodium Hypochlorite for further debridement.
5. Dispose of syringe and irrigating tip after use.

### Precautions:

1. Read SDS for material handling and safety information
2. Avoid allowing Sodium Hypochlorite to be ingested, use proper evacuation.
3. Avoid spilling on colored fabrics and surfaces.

### Storage:

Store in a cool, dry area away from heat and direct sunlight.



#### DANGER

Skin corrosion. Serious eye damage. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/eye protection/face protection/protective clothing. Harmful if swallowed. Wash thoroughly after handling and avoid any chemical contact with eyes, skin and clothing. Keep container tightly closed to avoid inhalation or accidental ingestion. Use with adequate ventilation when necessary. Follow all government regulations for waste disposal.



Discharge to the environment must be avoided. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Take off contaminated clothing and shoes immediately. Wash off with plenty of soap and water. Consult a physician. **IF INHALED:** Move person to fresh air. **IF SWALLOWED:** Do not induce vomiting. Rinse mouth with water. **Specific treatment:** Immediately call a Poison Control Center or doctor/physician. **FIRE: IN CASE OF FIRE:** Evacuate all personnel. Use dry powder, water spray, fog, mist, foam, sand or carbon dioxide to extinguish fire.



**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755  
www.vistaapex.com

**Rx ONLY**  
**Made in USA**

502300-I-V-ENG (7)



0459

**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
Tel: +31 (0)70 345 8570  
Fax: +31 (0)70 346 7299