VISTA APEX

**Chlor-XTRA™,  
6% NaOCl,  
3% NaOCl**

**REF 503800, 503825,  
317008, 502350,  
502300, 317018,  
502355, 502315**

**Rx ONLY**  

**Emergo Europe**  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755  
[www.vistaapex.com](http://www.vistaapex.com)

VISTA APEX

# Chlor-XTRA™, 6% NaOCl, 3% NaOCl

|   |    |
|---|----|
| FR - Solution de NaOCl à pour<br>l'irrigation canalaire ..... | 3  |
| EN - NaOCl solution for canal irrigation .....                | 4  |
| ES - Solución NaOCl al para irrigación<br>de canales .....    | 5  |
| DE - NaOCl ige Lösung zur<br>Kanalbewässerung .....           | 6  |
| IT - NaOCl soluzione al per<br>l'irrigazione dei canal .....  | 7  |
| PT - Solução de NaOCl para<br>irrigação de canais .....       | 8  |
| NL - NaOCl oplossing voor kanaalirrigatie .....               | 9  |
| PL - NaOCl roztwór do irygacji kanałów .....                  | 10 |
| EL - NaOCl διάλυμα για άρδευση καναλιών ..                    | 11 |
| BG - NaOCl разтвор за промиване<br>на канали.....             | 13 |
| CS - NaOCl roztok pro zavalážování kanálu ..                  | 14 |
| DA - NaOCl opløsning til kanalbanding .....                   | 15 |
| ET - NaOCl lahus kanalite niisutmiseks .....                  | 16 |
| HU - NaOCl -os oldat a<br>csatorna öntözésére .....           | 17 |
| LT - NaOCl skįidum kanálu apūdejošanai .....                  | 18 |
| LV - NaOCl tirpalas kanalu drēkinimui .....                   | 19 |
| RO - NaOCl soluție pentru<br>irigarea canalelor .....         | 20 |
| SK - NaOCl roztok na zavlažovanie kanálov ..                  | 21 |
| SL - NaOCl raztopina za<br>namakanje kanalov .....            | 22 |
| SV - NaOCl -lösning för avloppsebevättning ..                 | 23 |

**Description:**

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\*\*\*

**EN Description:**

NaOCl is an aqueous sodium hypochlorite solution. At low concentrations dissolve mainly necrotic tissue, at higher concentrations also dissolves vital tissue.

**Instruction for use:**

Irrigate with NaOCl as needed for debridement during instrumentation.

1. Fill syringe with NaOCl (10 mL recommended).
2. Attach irrigation tip securely to syringe.
3. Irrigate as desired.
4. Remove and dispose of syringe and irrigating tip.

**Precautions:**

1. Read MSDS for material handling and safety information.
2. Avoid spilling on surrounding tissues.
3. Avoid spilling on colored fabrics and surfaces as NaOCl is moderately alkaline and has a whitening effect.
4. Proper treatment planning and establishment of working length is required prior to use of this product.
5. Following standard of care, isolate the tooth prior to use of this product to prevent unintended tissue contact or accidental ingestion.
6. Properly evacuate the canal during use of this product to prevent unintended tissue contact or accidental ingestion.
7. It is recommended to use irrigation tips with lateral venting (27ga or smaller) to ensure effective distribution of irrigation solution throughout the root canal and minimize the risk of apical extrusion.

**Warnings:**

1. Caution, caustic!
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.
4. Over-instrumentation prior to use of this product can result in unintended tissue contact.
5. Under-instrumentation prior to use of this product can result in tip binding and subsequent apical extrusion. Do NOT bind the tip in the root canal.
6. If tip binding occurs during the procedure, or if resistance is felt during irrigation, discontinue irrigation until the tip binding issue is resolved.
7. NEVER apply excessive pressure when irrigating NaOCl.
8. If apical extrusion of NaOCl has occurred, seek medical attention immediately.
9. If ingested, do not induce vomiting. Drink a large portion of water and provide fresh air. Seek medical attention immediately.

**Storage:**

Store in a cool, dry area away from heat and direct sunlight.

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