



REF 502925 - Endo Kit

REF 503400 - 30mL Bulk Syringe Kit

REF 503930 - Endo Refill

REF 503455 - Pre-filled Syringes Refill

REF 503650 - 30mL Bulk Syringe

REF 503925 - Unit Dose

### Description:

CHX-Plus™ is an enhanced 2% Chlorhexidine Gluconate solution with powerful wetting agents and proprietary surface modifiers to improve depth of penetration. Used for the final irrigation step for long lasting cleansing.

### Instructions for Use:

1. Remove CHX-Plus™ syringe and irrigation tip from packaging.
2. Remove syringe cap and attach irrigation tip securely to syringe.  
For bulk (30mL) syringes, attach the provided docking port for transfer of solution to smaller syringes.
3. Irrigate as desired.
4. Dispose of syringe and irrigating tip after use.

### Warnings:

**\*\* NOT TO BE USED AS AN ORAL RINSE \*\***

- Irrigating tips are disposable
- Always replace cap after use



#### WARNING

May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation.



Wash hands thoroughly after handling. Wear protective gloves/eye glasses. Follow all government regulations for waste disposal. Prevent release to the environment if possible. Do not flush waste into sewer or waterways.

**FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water.



**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755  
www.vistaapex.com

**Rx ONLY**

**Made in USA**

502455-I-V-ENG (9)

EC REP



0459

**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
Fax: +31 (0)70 346 7299  
Tel: +31 (0)70 345 8570