VISTA I APEX

CHX
CHX-Plus

REF 317010, 503750
502450, 503900
502925, 503930
503650, 503400

Rx ONLY

2°C
36°F

30°C
86°F

MD

Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA

0459

EC **REP**

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in USA

502455-I-V-MUL(1)

VISTA I APEX

CHX, CHX-Plus

FR - CHX & CHX-Plus solution canaulaire	3	LV - CHX & CHX-Plus šaknīes kanālo tirpalas	10
EN - CHX & CHX-Plus root-canal solution	3	RO - CHX & CHX-Plus soluție de canal radicular	11
ES - CHX & CHX-Plus solución de conducto radicular	4	SK - CHX & CHX-Plus roztok koreňového kanálíka	11
DE - CHX & CHX-Plus Wurzelkanallösung	4	SL - CHX & CHX-Plus raztopina koreninskega kanala	12
IT - CHX & CHX-Plus soluzione al 2% per canale radicolare	5	SV - CHX & CHX-Plus root-canal lösning	12
PT - Solução CHX & CHX-Plus de canal radicular.....	5	502455-I-V-MUL(1)	
NL - CHX & CHX-Plus wortelkanaaloplossing	6		
PL CHX & CHX-Plus roztwór kanału korzeniowego	6		
EL - CHX & CHX-Plus διάλυμα ριζικού καναλιού	7		
BG - CHX & CHX-Plus разтвор на коренов канал	7		
CS - CHX & CHX-Plus roztok kořenového kanálu	8		
DA - CHX & CHX-Plus rodkanalopløsning	8		
ET - CHX & CHX-Plus juurekanali lahus	9		
HU - CHX & CHX-Plus gyökércsatorna-old.....	9		
LT - CHX & CHX-Plus sakņu kanāla risinājums	10		

(FR)

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(EN)**Description:**

CHX & CHX-Plu is a chlorhexidine digluconate solution. It is intended for the irrigation and debridement of root canals after instrumentation in endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12.5%.

For professional use only.

Instruction for use:

1. Use EDTA 17% as a pre-rinse in the canal to remove smear layer.
2. Rinse with Sodium Hypochlorite (NaOCl).
3. Rinse out the NaOCl with sterile water or anesthetic solution
4. Fill syringe with CHX & CHX-Plu (5-10 ml recommended). Attach irrigation tip securely to syringe.
5. Continuously irrigate for a minimum of 2 minutes. (10 – 20 ml per canal recommended).
6. Remove and dispose of syringe and irrigating tip.
7. Dry and obturate the root canal.

Precautions:

1. Read MSDS for material handling and safety information.
2. Do not ingest, do not inject.
3. Not to be used as an oral rinse.

Warnings:

1. Products containing chlorhexidine must not be mixed with products containing sodium hypochlorite to prevent formation of a reddish-brown low solubility precipitate.
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.

Storage:

Store at room temperature, dry area, away from heat and direct sunlight.

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