VISTA APEX

**CHX
CHX-Plus**

REF 317010, 503750
502450, 503900
502925, 503930
503650, 503400

Rx ONLY

Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Made in USA

502455-I-V-MUL (2)

CHX, CHX-Plus

FR - CHX & CHX-Plus solution canalaire	3
EN - CHX & CHX-Plus root-canal solution	3
ES - CHX & CHX-Plus solución de conducto radicular	4
DE - CHX & CHX-Plus Wurzelkanallösung	4
IT - CHX & CHX-Plus soluzione al 2% per canale radicolare	5
PT - Solução CHX & CHX-Plus de canal radicular.....	5
NL - CHX & CHX-Plus wortelkanaaloplossing	6
PL CHX & CHX-Plus roztwór kanału korzeniowego	6
EL - CHX & CHX-Plus διάλυμα ριζικού καναλιού	7
BG - CHX & CHX-Plus разтвор на коренов канал	7
CS - CHX & CHX-Plus roztok kořenového kanálu	8
DA - CHX & CHX-Plus rodkanalopløsning	8
ET - CHX & CHX-Plus juurekanali lahus	9
HU - CHX & CHX-Plus gyökércsatorna-old.....	9
LT - CHX & CHX-Plus sakņu kanāla risinājums	10
LV - CHX & CHX-Plus šaknies kanalo tirpalas	10
RO - CHX & CHX-Plus soluție de canal radicular	11
SK - CHX & CHX-Plus roztok koreňového kanálka	11
SL - CHX & CHX-Plus raztopina koreninskega kanala	12
SV - CHX & CHX-Plus root-canal lösning	12
502455-I-V-MUL(1)	

(FR)

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12-2021

(EN)**Description:**

CHX & CHX-Plus is a chlorhexidine digluconate solution. It is intended for the irrigation and debridement of root canals after instrumentation in endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12.5%.

For professional use only.

Instruction for use:

1. Use EDTA 17% as a pre-rinse in the canal to remove smear layer.
2. Rinse with Sodium Hypochlorite (NaOCl).
3. Rinse out the NaOCl with sterile water or anesthetic solution
4. Fill syringe with CHX & CHX-Plus (5-10 ml recommended). Attach irrigation tip securely to syringe.
5. Continuously irrigate for a minimum of 2 minutes. (10 – 20 ml per canal recommended).
6. Remove and dispose of syringe and irrigating tip.
7. Dry and obturate the root canal.

Precautions:

1. Read MSDS for material handling and safety information.
2. Do not ingest, do not inject.
3. Not to be used as an oral rinse.

Warnings:

1. Products containing chlorhexidine must not be mixed with products containing sodium hypochlorite to prevent formation of a reddish-brown low solubility precipitate.
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.

Storage:

Store at room temperature, dry area, away from heat and direct sunlight.

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