VISTA APEX

## Silane Refill

REF 502505

**Rx Only**

**Made in USA**

**CE**  
0459

**Emergo Europe**  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755

502505-I-V-MUL (2)

Do not use  
if package is  
damaged

- EN Do not use if package is damaged  
BG Не използвайте, ако опаковката е повредена  
CZ Nepoužívejte, pokud je obal poškozen  
DA Må ikke anvendes, hvis pakningen er beskadiget  
ET Ärge kasutage, kui pakend on kahjustatud  
FR Ne pas utiliser si l'emballage est endommagé  
DE Nicht verwenden, wenn die Verpackung beschädigt ist  
EL Μην το χρησιμοποιείτε εάν η συσκευασία είναι καταστραφείνη  
HU Ne használja, ha a csomagolás sérült  
IT Non utilizzare se la confezione è danneggiata  
LV Nelietot, ja iepakojums ir bojāts  
LT Nenaudokite, jei pakuotė pažeista  
PT Não use se o pacote estiver danificado  
RO Nu utilizați dacă ambalajul este deteriorat  
SK Nepoužívejte, ak je obal poškodený  
ES No utilice el producto si el envoltorio está dañado  
SV Får inte användas om förpackningen är skadad

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**SV** Frätande

**EN****Indications:**

Vista Porcelain Etch kit is to be used any time resin bonding to porcelain is required. Porcelain Etch is used for the preparation of porcelain crowns, veneers or inlays prior to cementation. It is also used in preparing fractured porcelain and porcelain-to-metal crown bonding.

**Instructions for Use:**

1. Isolate are to be etched with rubber dam.
2. Remove porcelain syringe and Spira-Flo™ tips from their packaging.
3. Remove syringe cap and attach tip securely to syringe.
4. If preparing fractured porcelain for bonding, first smooth off the surface of the porcelain being etched with a medium grit bur, removing the glazed surface. Rinse away all particles with water and dry prepared porcelain with moisture/oil-free dry air.
5. Apply Vista Porcelain Etch with brush tip to the prepared porcelain surface.
6. Let stand for 60 seconds, then rinse with water and dry surface (completely removing etchant).
7. Etched porcelain should appear frosty white. If not, repeat steps 5 & 6.
8. Apply Best-Etch to remove salts and debris (5 seconds), rinse and dry.
9. Remove Silane syringe and Spira-Flo™ Brush tip from packaging.
10. Remove syringe cap and attach tip securely to syringe.
11. Brush Vista Silane onto etched porcelain surface and let dry completely. If needed, dry with moisture/oil-free air.
12. Place bonding resin according to manufacturer's instructions.

**\*\*STORE AT ROOM TEMPERATURE\*\*****WARNING**

May cause eye, respiratory and skin irritation. Harmful if swallowed. Wear protective eye wear and gloves. Avoid unnecessary skin contact. May be disposed of in landfill or incinerator in accordance with local, state and federal regulations. **FIRST AID:**  
**IF ON SKIN:** Wash with plenty of soap and water.  
**IF IN EYES:** Flush with large quantities of water for fifteen (15) minutes. Consult a physician.  
**IF SWALLOWED:** Drink two glasses of water or milk. Seek medical attention if a substantial amount has been ingested. Do not induce vomiting.

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