



REF 502700

REF 502755

REF 502705

REF 503450

REF 502750

REF 503500

## Indications

*Vista Caries Indicator is used to:*

- Identify carious dentin
- Ensure the maximum amount of inner dentin is preserved and infected dentin is removed.
- Easily locate root canal orifice and calcified canals

## Instructions for Use:

1. Remove Caries Indicator syringe and Sol-u-Flo™ brush tip from packaging.
2. Remove syringe cap and attach brush tip securely to syringe.
3. Apply Caries Indicator with brush tip to tooth.
4. Wait 5-10 seconds and rinse with water.
5. Identify carious dentin.
6. Remove and dispose of brush tip from the syringe.
7. Clean and disinfect syringe between patients.

## Storage:

- Refrigeration is not required, store at room temperature (22° C / 72°F).
- Do not store in direct sunlight or extreme heat.



### WARNING

Causes irritation to eyes and skin. May cause nausea, headache and gastrointestinal disturbances.

Wear safety goggles and appropriate gloves. Allergic skin reaction is possible. Waste may be considered as inert material.

Dispose of safely in accordance with local, state and federal regulations. **FIRST AID: IF INHALED:** Exit to fresh air.

**IF INGESTED:** Rinse mouth and seek immediate medical advice. **IF ON SKIN:** Wash affected area with soap and water.

**IF IN EYES:** Rinse immediately with plenty of water and seek medical advice.



**Inter-Med, Inc.**

2200 South St.

Racine, WI 53404 USA

Tel: +1-262-636-9755

www.vistaapex.com

**Rx ONLY**

**Made in USA**

502700-I-V-ENG (5)



**Emergo Europe**

Prinsessegracht 20

2514 AP The Hague

The Netherlands

Tel: +31(0)70 345 8570

Fax: +31(0)70 346 7299

