



Vista Caries Indicator

REF 502700 -
Standard Kit, Red

REF 502705 -
Refill Pack (4pk), Red

REF 503450 -
Value Pack (20pk), Red

REF 502750 -
Standard Kit, Green

REF 502755 -
Refill Pack (4pk), Green

REF 503500 -
Value Pack (20pk), Green

Indications:

Vista Caries Indicator is used to:

- Identify carious dentin
- Ensure the maximum amount of inner dentin is preserved and infected dentin is removed.
- Easily locate root canal orifice and calcified canals.

Instructions for Use:

1. Remove Caries Indicator syringe and Sol-u-Flo™ brush tip from packaging.
2. Remove syringe cap and attach brush tip securely to syringe.
3. Apply Caries Indicator with brush tip to tooth.
4. Wait 5-10 seconds and rinse with water.
5. Identify carious dentin.
6. Remove and dispose of brush tip from the syringe.
7. Clean and disinfect syringe between patients.

Storage:

- Refrigeration is not required, store at room temperature (22° C/ 72°F).
- Protect from freezing.
- Do not store in direct sunlight or extreme heat.

NOTE: See vistaapex.com for SDS.



WARNING

Causes irritation to eyes and skin. May cause nausea, headache and gastrointestinal disturbances. Wear safety goggles and appropriate gloves. Allergic skin reaction is possible. Waste may be considered as inert material. Dispose of safely in accordance with local, state and federal regulations. **FIRST AID: IF INHALED:** Exit to fresh air. **IF INGESTED:** Rinse mouth and seek immediate medical advice. **IF ON SKIN:** Wash affected area with soap and water. **IF IN EYES:** Rinse immediately with plenty of water and seek medical advice.



Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755
www.vistaapex.com



Rx ONLY

Made in USA
502700-I-V-ENG (7)