



VISTA APEX

Vista-BLUE™

REF 503429 - Standard Kit



Rx ONLY

Harmful if swallowed. Avoid contact with skin and eyes.

Description:

Vista-BLUE™ is a water soluble solution which is used for the identification of cracks, calcified canals, root canal orifices and carious dentin.

Method:

1. Remove Vista-BLUE™ bottle, mixing well, and Micro-Applicator brushes from packaging.
2. Remove bottle cap and dispense desired amount of Vista-BLUE™ into mixing well.
3. Saturate Micro-Applicator brush tip with Vista-BLUE™.
4. Apply Vista-BLUE™ to tooth.
5. Wait 5 – 10 seconds and rinse with water.
6. Identify the fracture.
7. Reapply as needed.
8. Dispose of Micro-Applicator brush after each patient.

Storage:

1. Store refrigerated. Protect from freezing.
2. Avoid contact with skin, eyes and clothing.
3. In the event of eye contact, flush with copious amounts of water and seek immediate medical attention.
4. If contact with clothing occurs, stain may be removed with mild detergent and warm water.

NOTE: See vistaapex.com for SDS.



Inter-Med, Inc.

2200 South St., Racine, WI 53404, USA

Tel: +1-262-636-9755

www.vistaapex.com

Made in USA
503429-I-V-ENG (5)