



Descriptions:

CHX is a 2% Chlorhexidine Gluconate Solution in water. CHX is a root canal cleanser for use in endodontic procedures. After endodontic instrumentation, these products should be used to cleanse the canal space before placement of the endodontic filling.

Indications:

CHX is used as a final endodontic rinse after instrumentation to irrigate and cleanse the root canal system for long lasting cleansing.

Instructions for Use:

1. For 1.2ml syringe
 - a. Remove luer cap.
 - b. Securely attach tip.
 - c. Verify flow prior to intraoral use.
2. For 30mL Prefilled Syringe
 - a. Remove luer cap from 30mL prefilled syringe.
 - b. Attach one end of the docking port to the 30mL prefilled syringe and attach the other end of the docking port to a 1.2mL syringe.
 - c. Slowly depress 30mL prefilled syringe plunger while guiding 1.2ml syringe plunger to desired fill.
 - d. Separate syringes from the docking port and re-cap 30mL prefilled syringe.
 - e. Securely attach tip to 1.2mL syringe.
 - f. Verify flow prior to intraoral use.
 - g. Clean and disinfect 30mL prefilled syringe between patients.
3. Once canal has been cleaned and shaped to appropriate size, rinse with CHX.
4. Air dry.
5. Obturate with materials of choice.
6. Dispose of tip and unit dose syringe.

Warnings:

** NOT TO BE USED AS AN ORAL RINSE **

- Tips are disposable. Do not reuse.
- To avoid clogging, always replace cap after use.
- Avoid contact with skin and eyes.
- Do not ingest material.
- Do not use on patients with known sensitivity to chlorhexidine.
- Read SDS for material handling and safety information.



WARNING

May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation.

Wash hands thoroughly after handling. Wear protective gloves/eye glasses. Follow all government regulations for waste disposal. Prevent release to the environment if possible. Do not flush waste into sewer or waterways.

FIRST AID: **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water.



Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755
www.vistaapex.com

Rx ONLY
Made in USA
503750-I-V-ENG (8)



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

