

Description:

Vista's Desensitizing Gel is a potassium nitrate and fluoride gel. Desensitizing Gel is designed to be used with Fluorescent™ whitening systems.

Indication:

Quickly eliminates hypersensitivity associated with whitening, temperate and chemical changes. Desensitizing Gel should be used by a Dental Professional.

Precautions:

1. Use a rubber dam or other suitable isolation method to prevent contamination of surfaces.
2. Keep Desensitizing Gel away from small children.
3. Not intended for use by pregnant women.
4. **NOT** for nighttime usage.
5. Do not use tobacco products or eat while using this product.
6. If problems occur, discontinue usage and notify dentist.
7. Keep out of heat and/or sunlight at all times. Gel should be stored at room temperature.
8. Patients should not swallow large amounts of gel. Excess material should be brushed off and rinsed. **DO NOT** swallow.

Instructions for Use:

In-Office Application:

1. Rinse the tooth surface and dry. Avoid cold water or compressed air.
2. Apply Desensitizing Gel to sensitive surface for about 20-40 seconds.
3. Wipe away minor excess gel from the tooth with a cotton roll or applicator.
4. Test the surface with a gentle stream of air for 5-10 seconds.
If the patients sensitivity persists, repeat the application.
5. Discard applicator tip and recap the syringe if material is left to prevent drying out.

Take-Home Application:

1. Fabricate tray to meet patient's needs. Areas of sensitivity should be indicated before sending the model. If patient already has a tray, the same one can be used.

2. Desensitizing Gel is dispensed in 1.2mL dose syringes. A complete arch will use most of the contents. If sensitivity is localized, place a small amount in affected area(s) and a small amount in the molar region to hold in place.
3. Recommended treatment times can range from 15 minutes to 1 hour. The length and number of times tray is worn depends on condition, patient and clinician recommendation.



WARNING

Causes serious eye irritation.

Wear eye/face protection. Prolonged or repeated skin contact may cause severe irritation.

FIRST AID:

IF IN EYES: Rinse cautiously with water for several minutes, also under eye lids. Remove contact lenses, if present and easy to do. **IF INHALED:** Remove to fresh air and keep at a rest position comfortable for breathing. **IF INGESTED:** Do not induce vomiting. Rinse mouth, drink plenty of water. Call a physician or Poison Control Center immediately.

Rx ONLY

CE
0086



Inter-Med, Inc.
2200 South St.,
Racine, WI 53404
Tel: +1-262-636-9755

Made in USA

504292-I-V-71-ENG (1)



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands