

**KEEP CONTENTS  
REFRIGERATED**

Bring to room temperature  
before using

**Kit includes:**

- (1) Pre-filled Chlor-XTRA™ syringe – 12cc
- (1) Pre-filled SmearOFF™ syringe – 6cc
- (2) 30ga Appli-Vac™ Irrigating tips
- (1) Micro Aspirator with tip

**Instructions for Use:**

**#1 Chlor-XTRA™  
(Sodium Hypochlorite)**

To debride/clean canal during  
and after instrumentation

1. Attach irrigating tip  
securely onto the syringe.
2. Irrigate canals with Chlor-XTRA™.
3. Dispose of syringe and  
irrigating tip after use.

**#2 SmearOFF™**

Removes smear layer /  
Final irrigation step for long  
lasting cleansing

1. Attach irrigating tip  
securely onto the syringe.
2. Irrigate canals with SmearOFF™.
3. Dispose of syringe and  
irrigating tip after use.

**Repeat Steps 1-2**

**LIST OF REFERENCES**

**Chlor-XTRA™:** Williamson AE, Cardon JW, Drake Dr. Antimicrobial susceptibility of monoculture biofilms of a clinical isolate of *Enterococcus faecalis*. *J Endod.* 2009 Jan;35(1):95-7.

Palazzi F, Morra M, Mohammadi Z, Grandini S, Giardino L. Comparison of the surface tension of 5.25% sodium hypochlorite solution with three new sodium hypochlorite-based endodontic irrigants. *Int Endod J.* 2011 Sep 11.

Stojicic S, Zivkovic S, Qian W, Zhang H, Haapasalo M. Tissue dissolution by sodium hypochlorite: effect of concentration, temperature, agitation, and surfactant. *J Endod.* 2010 Sep; 36(9):1558-62.

**CHLOR-XTRA™**



**DANGER**

Skin corrosion. Serious eye damage. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/eye protection/face protection/protective clothing. Harmful if swallowed. Wash thoroughly after handling to avoid any chemical contact with eyes, skin and clothing. Keep container tightly closed to avoid inhalation or accidental ingestion. Use with adequate ventilation when necessary. Store in a tightly closed container. Follow all government regulations for waste disposal. Do not flush product into sewer or drains that may lead to waterways. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Take off contaminated clothing and shoes immediately. Wash off with soap and plenty of water. **IF INHALED:** Move person to fresh air. **IF SWALLOWED:** Do not induce vomiting. Rinse mouth with water. **SPECIFIC TREATMENT:** Contact a POISON CONTROL CENTER or doctor/physician. **FIRE: IN CASE OF FIRE:** Evacuate all personnel. Use extinguishing media most appropriate for the surrounding fire. Substance is noncombustible. Nonflammable; contact with metals may evolve flammable hydrogen gas.

**SMEAROFF™**



**WARNING**

May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation. Wash hands thoroughly after handling. Wear protective gloves/eye wear/face wear. Dispose of contents in accordance with local, state, and federal regulations. **FIRST AID: IF ON SKIN:** Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. Call a poison control center or doctor/physician if you feel unwell. If skin irritation occurs or eye irritation persists, get medical advice/attention.



**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755

**Made in USA**

504700-I-V-09-ENG (3)