VISTA APEX

**Disposable
MTA Carrier**

REF 807015, 807016,
807017, 807019

Rx Only

CE
0459

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755

807016-I-V-MUL (3)

Do not use
if package is
damaged

- EN** Do not use if package is damaged
BG Не използвайте, ако опаковката е повредена
CZ Nepoužívejte, pokud je obal poškozen
DA Må ikke anvendes, hvis pakningen er beskadiget
ET Ärge kasutage, kui pakend on kahjustatud
FR Ne pas utiliser si l'emballage est endommagé
DE Nicht verwenden, wenn die Verpackung beschädigt ist
EL Μην το χρησιμοποιείτε εάν η συσκευασία είναι καταστραμμένη
HU Ne használja, ha a csomagolás sérült
IT Non utilizzare se la confezione è danneggiata
LV Nelietot, ja iepakojums ir bojāts
LT Nenaudokite, jei pakuotė pažeista
PT Não use se o pacote estiver danificado
RO Nu utilizați dacă ambalajul este deteriorat
SK Nepoužívajte, ak je obal poškodený
ES No utilice el producto si el envoltorio está dañado
SV Får inte användas om förpackningen är skadad

Manufacturer

- EN** Manufacturer
BG Производител
CZ Výrobce
DA Producent
ET Tootja
FR Fabricant
DE Hersteller
EL Κατασκευαστής
HU Gyártó
IT Produttore
LV Ražotājs
LT Gamintojas
PT Fabricante
RO Producător
SK Výrobca
ES Fabricante
SV Tillverkare

Do Not Reuse

- EN** Do not reuse
BG Не използвайте повторно
CZ Nepoužívejte znovu
DA Ikke til flergangsbrug
ET Ärge taaskasutage
FR Ne pas réutiliser
DE Nicht wiederverwenden
EL Ärge taaskasutage
HU Ne használja újra
IT Non riutilizzare
LV Nelietot atkārtoti
LT Nenaudoti pakartotinai
PT Não reutilize
RO Nu reutilizați
SK Nepoužívajte opakovane
ES No reutilizar
SV Får inte återanvändas

Rx Only

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EN**Indications for Use:**

Device is intended to deliver MTA to the root canal during endodontic procedures.

Instructions for Use - Mixing MTA:

MTA should be mixed to a consistency similar to "wet sand". It typically needs to be just wet enough to hold an impression of the tube when packed. This should form a small pellet of material when delivered.

- 1.) Remove Disposable MTA Carrier from pouch and place in autoclave (can be placed in autoclavable pouch). NOTE: Autoclave at 132°C (270°F) - 135°C (275°F) for 3-4 minutes.
- 2.) Once autoclaved, retract plunger until it hits the lock.
- 3.) Tap open end of carrier into the MTA mixture, drawing it into cannula.

1.6mm	2-3 taps*
0.99mm	2-6 taps*
0.80mm	2-10 taps*

* Number of taps may vary based on pressure applied, amount of material in well, and material consistency.

During the procedure:

MTA can be delivered by pressing the plunger down. When mixed properly, a small amount of material will be delivered. If plunger "sticks", apply ultrasonic energy to the carrier while attempting to deliver material. When not in use for a short period of time during the procedure, dip the tip of the carrier into sterile water or saline between placements. If prolonged period is anticipated between clinical uses on the same patient, rinse bore or inner tube with sterile water using a syringe.

The entire carrier should be discarded after use on each patient.

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