



CLEAN & BOOST[®]
DENTIN & ENAMEL CLEANSER




REF 90056, 90057, 90058, 90417

Rx Only



Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755
www.vistaapex.com

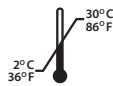

NOTE: Visit vistaapex.com for SDS

90377-I-AP-ENG (6)

DESCRIPTION:

Clean & Boost[®] when utilized properly, will remove any and all contaminants that could negatively impact adhesion. Along with cleansing the surface, Clean & Boost[®] will etch tooth surfaces to increase bond strengths of the self-etch materials with inherently weak etching capabilities.

Clean & Boost[®] is completely rinsed from the surface after use and thus is compatible with all other dental materials.

INDICATIONS FOR USE:

Clean & Boost[®] is an acidic, aqueous cleanser that has been designed to help remove contaminants from the surface of a tooth prior to bonding. These contaminants can be hand piece oil, tooth debris, or imaging powder (used to create digital impressions for CAD/CAM restorations).

PRINCIPAL USES OF CLEAN & BOOST[®]:

- Cleansing dentin and enamel surfaces of all contaminants
- Removing imaging powder from all surfaces
- Disinfecting tooth surfaces
- Etching tooth structure for improved adhesion and long term results

INSTRUCTIONS FOR USE:

1. With the enclosed delivery brush tip, apply Clean & Boost[®] to the surface to be cleansed. Surface should be completely saturated with Clean & Boost[®].
2. Using the delivery brush tip, agitate the Clean & Boost[®] on the surface for 10 seconds.
3. Rinse thoroughly.

STORAGE:

Protect from sunlight. Store between 2°C/36°F and 30°C/86°F. Refrigerate materials that are not to be used for more than 7 days. See individual component labels for specific expiration dates.

CAUTIONS:

Avoid contact with eyes. If the material comes into contact with the eyes, flush with copious amounts of water and seek medical attention. Un-polymerized resins may cause skin sensitization in susceptible persons, in case of contact with the skin, wash thoroughly with soap and water.