VISTA | APEX

interface

REF *90030, SDL19, SDL20

Rx Only

* Asterisk notes CE marked SKU

Emergo Europe

Prinsessegracht 20
2514 AP The Hague
The Netherlands

Inter-Med, Inc.

2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755

www.vistaapex.com

90382-I-AP-MUL (1)

EN Catalogue number
FR Référence catalogue
DE Bestellnummer
IT Numero di catalogo
ES Número de catálogo

**Caution,
Consult
Accompanying
Documents**

EN Caution, consult accompanying documents
FR Attention, consulter les documents joints
DE Vorsicht, Begleitdokumente beachten
IT Attenzione, consultare la documentazione allegata
ES Precaución: consulte documentos adjuntos

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**Authorized
Representative
in the European
Community**

EN Authorized Representative in the European Community
FR Représentant agréé pour la Communauté européenne
DE Autorisierte Vertretung in der Europäischen Gemeinschaft
IT Rappresentante autorizzato nella Comunità Europea
ES Representante autorizado en la Comunidad Europea

Manufacturer

EN Manufacturer
FR Fabricant
DE Hersteller
IT Produttore
ES Fabricante

Rx Only

EN CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.
FR ATTENTION: la loi fédérale américaine stipule que la vente de ce dispositif est réservée aux médecins ou sur prescription médicale.
DE VORSICHT: US-Bundesgesetz beschränkt den Verkauf dieses Produkts an einen Arzt oder auf dessen Anordnung.
IT ATTENZIONE: la legge federale statunitense limita la vendita di questo dispositivo a medici o su prescrizione medica.
ES PRECAUCIÓN: las leyes federales de los Estados Unidos solo permiten la venta de este dispositivo por parte un médico o por prescripción facultativa.

EN**Description:**

Interface is a revolutionary ceramic primer, which allows the clinician to bond any type of ceramic to a tooth, including the newer high-strength materials. With Interface, any ceramic restoration can be delivered or repaired (in vivo) without the use of dangerous hydrofluoric acid. Since it is activated when it is used, the shelf life of the primer is extremely long (3 years from the date of manufacture). Perhaps best of all, Interface can be used to prepare ceramic, dentin and enamel all at the same time.

Indications for Use:

Interface is a ceramic primer/conditioner, which allows the clinician to bond any type of ceramic (including newer high strength ceramic materials) to a tooth. Interface when used in conjunction with a resin adhesive can be used for ceramic and ceramic to tooth repairs, as well as bonding ceramic inlays, onlays, crowns and veneers. Interface is to be used as a ceramic primer/conditioner allowing bonding to ceramic, dentin and enamel all at the same time. With Interface, any dental ceramic can be repaired safely in the mouth without the use of dangerous hydrofluoric acid.

General Instructions:**Preparation of Interface**

Interface is furnished in two bottles, labeled A and B, along with a mixing well and Microtip® applicators.

1. A drop of Interface A and a drop of Interface B are dispensed into the same cavity of the mixing well. At first the two liquids will not mix – one will form a “bubble” on the top of the other.
2. After 20-30 seconds, the bubble will break down, and at that time the mixture is stirred for 5 seconds with the Microtip® applicator. Note: The mixed Interface solution must be used within 3 minutes. Once it has become cloudy or opaque it is expired and should be discarded.

Delivery of Interface

It is highly recommended that the ceramic surfaces to be bonded first are lightly sandblasted.

1. Using a Microtip® applicator, apply an even coating of the prepared Interface solution to the ceramic surface and allow to dwell for 10 seconds.
2. Dry the Interface solution thoroughly (approximately 5 seconds).

The surface has now been etched and silanated.¹

Delivery of Ceramic Inlays, Onlays, Crowns and Veneers²

1. Using a Microtip® applicator, apply an even coating of the prepared Interface solution to the ceramic surface and allow to dwell for 10 seconds.
2. Dry the Interface solution thoroughly (approximately 5 seconds).
3. If using **Simplicity**:
 - a. Apply 1 coat of Simplicity 2 to the dried Interface surface
 - b. Dry thoroughly
 - c. Light activate for 10 seconds
 - d. Apply a final coat of Simplicity 2.
 - e. Dry thoroughly but do not light activate

If using **Surpass**®:

- a. Apply 2 coats of Surpass® 2 and dry thoroughly
 - b. Light activate for 10 seconds
4. Deliver the prepared restoration with Anchor® Cement or material of your choosing.

Repairing of Ceramics²

1. Using a Microtip® applicator, apply an even coating of the prepared Interface solution to the ceramic surface and allow to dwell for 10 seconds.
2. Dry the Interface solution thoroughly (approximately 5 seconds).
3. If using **Simplicity**:
 - a. Apply 1 coat of Simplicity 2 to the dried Interface surface
 - b. Dry thoroughly
 - c. Light activate for 10 seconds

If using **Surpass**[®]:

- a. Apply 2 coats of Surpass[®] 2 and dry thoroughly
 - b. Apply 1 coat of Surpass[®] 3
 - c. Air thin (if desired)
 - d. Light activate for 10 seconds
4. Composite resin is then added and finished according to manufacturer's instructions.

Repairing Ceramic and Tooth (*Simplicity or Surpass*[®] is **REQUIRED** for this process)

1. Using a Microtip[®] applicator, apply an even coating of the prepared Interface solution to the ceramic surface and allow to dwell for 10 seconds.
2. Dry the Interface solution thoroughly (approximately 5 seconds)
3. Apply the prepared Interface solution to the exposed tooth structure evenly and agitated gently for 10 seconds.

DO NOT DRY THE INTERFACE SOLUTION ON THE TOOTH.

Note: Excess Interface solution on the ceramic surface will not adversely affect the bond strength of the repair.

4. If using **Simplicity**:
 - a. Apply 3 coats of Simplicity 2 to the Interface surface on both the tooth and ceramic
 - b. Dry thoroughly
 - c. Apply 2 additional coats of Simplicity 2
 - d. Dry thoroughly
 - e. Light activate for 10 seconds

If using **Surpass**[®]:

- a. Apply 3 coats of Surpass[®] 2 to the Interface surface on both the tooth and ceramic
 - b. Dry thoroughly
 - c. Apply 1 coat of Surpass[®] 3. Air thin (if desired)
 - d. Light activate for 10 seconds
5. Composite is then added and finished according to manufacturer's instructions.

Repair of Porcelain-to-metal Crowns

1. Roughen the surface to be repaired with a diamond.

2. Using a Microtip[®] applicator, apply an even coating of the prepared Interface solution to the ceramic surface and allow to dwell for 10 seconds.
 3. Dry the Interface solution thoroughly (approximately 5 seconds).
 4. If using **Simplicity**:
 - a. Apply 1 coat of Simplicity 2 to the dried Interface surface
 - b. Dry thoroughly
 - c. Light activate for 10 seconds
- If using **Surpass**[®]:
- a. Apply 2 coats of Surpass[®] 2 and dry thoroughly
 - b. Apply 1 coat of Surpass[®] 3
 - c. Air thin (if desired)
 - d. Light activate for 10 seconds
5. Composite resin is then added and finished according to manufacturer's instructions.

Storage:

Protect from sunlight. Store between 2°C/36°F and 30°C/86°F. Refrigerate materials that are not to be used for more than 7 days. See individual component labels for specific expiration dates. Do not use after the expiration dates.

Cautions:

Avoid contact with eyes. If the material comes into contact with the eyes, flush with copious amounts of water and seek medical attention. Un-polymerized resins may cause skin sensitization in susceptible persons, in case of contact with the skin, wash thoroughly with soap and water.

Notes:

Visit vistaapex.com for SDS.

¹ Vista Apex recommends the use of a resin based bonding agent such as Simplicity or Surpass[®] to complete the ceramic preparation prior to cementation. Detailed instructions included in this IFU.

² If using a different adhesive or self etching cement, follow manufacturer's instructions.

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