




**REF 90492, 90493, 90494,  
90495, 90047 kit, 90048 kit**

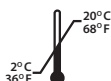

**Rx Only**



**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755  
[www.vistaapex.com](http://www.vistaapex.com)

**NOTE:** Visit [vistaapex.com](http://vistaapex.com) for SDS

90383-I-AP-ENG (5)

## DESCRIPTION:

Anchor® is an innovative core build-up material with dual cure chemistry packaged in an automix cartridge or an automix push syringe. Anchor® is offered in both tooth shade (A2) and opaque white. Anchor® has been designed to be thin enough for use as a luting material while having the strength and thixotropic characteristics required for core applications. Anchor® is a dual-cure material so you are assured the material will completely cure in any situation. Anchor® can be used as a core build-up material, cement for pins or posts and luting agent for crowns, veneers, etc. The chemistry is compatible with common composites, compomers and their bonding agents. Anchor® is extremely radiopaque and contains fluoride.

## INDICATIONS FOR USE:

- Core Build Ups
- Cementing Posts and Pins
- Cementing Crowns
- Cementing Veneers
- Cementing Inlays and Onlays
- Cementing CAD/CAM Restorations
- Cementing CEREC® Restorations

## GENERAL INSTRUCTIONS<sup>1,2,3,4,5,6:</sup>

For optimal results the tooth should be prepared with a resin based adhesive, such as SURPASS® Self Etch Bonding System or Superb Universal Adhesive prior to the use of Anchor®. Anchor® is compatible with all resin based adhesives and composites):

### 25ml Cartridge Assembly:

1. Load the dispensing gun with the automix 1:1 cartridge.
2. Remove and dispose of the cap. **DO NOT REUSE THE CAP.**
3. Extrude a pea-sized amount of material until both sides flow equally. (This only needs to be done the first time the syringe is used.)
4. Attach the mixing tip to the cartridge.
5. Extrude material as needed<sup>1</sup> For core build-ups, it is recommended to start the build up by applying a 1mm increment of Anchor® and light curing for 20 seconds. After this initial layer has been placed additional material should be used as necessary.

After each use, leave the used mixing tip in place to seal the cartridge.

### 5ml Syringe Assembly:

1. Twist the cap of the 5ml syringe and remove the cap. Be sure to discard the cap and do not use the cap again.
2. Depress the plunger to extrude a very small amount of the material (less than a pea size drop). This will insure that both sides flow properly. (This only needs to be done the first time the syringe is used.)
3. Insert the special mixing tip onto the syringe and secure with a twisting motion.

## GENERAL INSTRUCTIONS<sup>1,2,3,4,5,6</sup> (continued):

4. Extrude the material as needed<sup>1</sup>. For core build-ups, it is recommended to start the build up by applying a 1mm increment of Anchor® and light curing for 20 seconds. After this initial layer has been placed additional material should be used as necessary.

After each use, leave the used mixing tip in place to seal the syringe.

## ANCHOR® INSTRUCTIONS (WITH SURPASS®):

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air.
2. Dispense a drop of SURPASS® 1 into one cavity of the enclosed mixing well. Using the proper color-coded brush, apply SURPASS® 1 to the entire cavity preparation and the surrounding tooth structure, agitating briskly for at least 10 seconds. **DO NOT DRY SURPASS® 1**
3. Dispense a drop of SURPASS® 2 into another cavity of the mixing well. Using the proper color-coded brush apply three brushfuls of SURPASS® 2 to the entire cavity preparation, dabbing each brushful to help distribute as much of the primer as possible.
4. Dry the primed surface with dried compressed air for at least 10 seconds with the air syringe tip as close as possible to the surface. **DRY SURPASS® 2 THOROUGHLY**
5. Dispense a drop of SURPASS® 3 into another cavity of the mixing well. Using the proper color-coded brush apply a thin layer of SURPASS® 3 to the entire cavity surface. Air-thin the SURPASS® 3 aggressively. Final film thickness will be around 10 – 12 microns.
6. Light-activate for 10 seconds. SURPASS® can be activated with any dental activation light.

## 25ml Cartridge Assembly<sup>1</sup>:

7. Load the dispensing gun with the automix 1:1 cartridge.
8. Remove and dispose of the cap. **DO NOT REUSE THE CAP.**
9. Extrude a pea-sized amount of material until both sides flow equally. (This only needs to be done the first time the syringe is used.)
10. Attach the mixing tip to the cartridge.
11. Extrude material as needed. *For core build-ups, it is recommended to start the build up by applying a 1mm increment of Anchor® and light curing for 20 seconds. After this initial layer has been placed additional material should be used as necessary.*

After each use, leave the used mixing tip in place to seal the cartridge.

## 5ml Syringe Assembly:

7. Twist the cap of the 5ml syringe and remove the cap. Be sure to discard the cap and do not use the cap again.
8. Depress the plunger to extrude a very small amount of the material (less than a pea size drop). This will insure that both sides flow properly. (This only needs to be done the first time the syringe is used.)

## ANCHOR® INSTRUCTIONS (WITH SURPASS®)

### 5ml Syringe Assembly (*continued*):

9. Insert the special mixing tip onto the syringe and secure with a twisting motion.
10. Extrude the material as needed. *For core build-ups, it is recommended to start the build up by applying a 1mm increment of Anchor® and light curing for 20 seconds. After this initial layer has been placed additional material should be used as necessary.*

After each use, leave the used mixing tip in place to seal the syringe.

### STORAGE:

Protect from sunlight. Store between 2°C/36°F and 20°C/68°F.

Refrigerate materials that are not to be used for more than 7 days.

See individual component labels for specific expiration dates. Do not use after the expiration dates.

### CAUTIONS:

Avoid contact with eyes. If the material comes into contact with the eyes, flush with copious amounts of water and seek medical attention.

Un-polymerized resins may cause skin sensitization in susceptible persons, in case of contact with the skin, wash thoroughly with soap and water.

### NOTES:

1. Anchor® has been designed to have a working time of 1.5 minutes and fully cure in 4 minutes when used only in self-cure mode. (The setting time will be slightly less than 4 minutes in the mouth). If you use a dental curing light you can cure for 1 to 3 seconds to achieve a partial set stage and allow Anchor® to complete the curing process in the self-cure mode for an additional 3 minutes. The unique dual cure feature allows you to place the material and 'tack' it into place for 3 seconds, shape the restoration and continue while the self-cure feature completes the curing of the material. If used only in the light cure mode, total curing time will be 40 seconds. As with all light cured products, times will vary with the intensity of the curing light.
2. For the easiest clean up, wait for the 90 second working time to expire and the material to gel. The gel state can also be attained by light activating for 1-2 seconds. Using an explorer, remove excess material from the margins and floss between the contacts.
3. Shape before final curing using instruments, matrix bands or crown forms. Remove matrix band if used. Final crown preparation can be accomplished with your preferred procedures.
4. If using a post or a pin, prepare and insert the post or pin according to the manufacturer's directions.
5. Material can be extruded on pad for hand placement if desired
6. Visit [vistaapex.com](http://vistaapex.com) for SDS.