



**SUPERB**  
UNIVERSAL ADHESIVE




**REF** **SDL17, SDL18**

**Rx Only**



**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755  
[www.vistaapex.com](http://www.vistaapex.com)

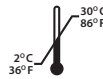

30°C  
86°F  
2°C  
36°F  
Protect From  
Freezing

**NOTE:** Visit [vistaapex.com](http://vistaapex.com) for SDS

90384-I-AP-ENG (7)

## **DESCRIPTION:**

SUPERB Universal Adhesive is a single bottle bonding system for use in ALL dental restorations. SUPERB Universal Adhesive combines the etchant, primer and bonding resin into a single component. Through the elimination of components and steps within the application protocol, SUPERB Universal Adhesive is extremely easy to apply and has little to no technique sensitivity. SUPERB Universal Adhesive provides: easy application, consistent results, best in class bonding and the elimination of post operative sensitivity in a single bottle. SUPERB Universal Adhesive can be used in 3 modes: Self Etch, Selective Etch or Total Etch. All 3 techniques will provide exceptional results.

## **INDICATIONS FOR USE:**

- Direct Restorations: to seal enamel / dentin prior to restoring with light-cure or self-cure composite material
- Indirect Restorations: bonding light-cured, self-cured or dual-cured composite cements or glass ionomer or resin-based ionomer cements
- Bonding CEREC® restorations
- Desensitization: to treat hypersensitive and / or exposed root surfaces
- Desensitizing crown preparations
- Sealing preparations before placing amalgams
- Bonding crowns and bridges
- Bonding in posts and cores
- Bonding veneers

## **GENERAL INSTRUCTIONS<sup>1,2,3,4</sup>:**

### MODE: SELF ETCH

1. Rinse the preparation thoroughly and remove any pooled water. Leave tooth moist.
2. Dispense a drop of SUPERB Universal Adhesive into the cavity of the enclosed mixing well. Using a bristle brush applicator, apply 1 coat of SUPERB Universal Adhesive to the entire cavity preparation and the surrounding tooth structure, allow material to dwell for twenty seconds. DO NOT DRY.
3. Using the remaining material in the well and same applicator, apply 2 additional coats of SUPERB Universal Adhesive
4. Dry the surface thoroughly. Do not blast the surface with air or use suction as this could remove material from the surface and lead to debonds or post operative sensitivity.
5. Light-activate for ten seconds. SUPERB Universal Adhesive can be activated with any dental activation light.
6. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

## MODE: SELECTIVE ETCH

Accidental etching of dentin will not negatively impact bond strengths.

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air. Do not desiccate the tooth surface.
2. Apply Clean & Boost® (Gel or Liquid) to the enamel surfaces to be bonded. Agitate for ten seconds, rinse thoroughly and leave moist.
3. Dispense a drop of SUPERB Universal Adhesive into the cavity of the enclosed mixing well. Using a bristle brush applicator, apply 1 coat of SUPERB Universal Adhesive to the entire cavity preparation and the surrounding tooth structure, allow material to dwell for twenty seconds. DO NOT DRY.
4. Using the remaining material in the well and same applicator, apply 2 additional coats of SUPERB Universal Adhesive.
5. Dry the surface thoroughly. Do not blast the surface with air or use suction as this could remove material from the surface and lead to debonds or post operative sensitivity.
6. Light-activate for ten seconds. SUPERB Universal Adhesive can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

## MODE: TOTAL ETCH

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air. Do not desiccate the tooth surface.
2. Apply Clean & Boost® (Gel or Liquid) to the tooth surface to be bonded (both dentin and enamel). Agitate the material for 10 seconds , rinse thoroughly and leave moist.
3. Dispense a drop of SUPERB Universal Adhesive into the cavity of the enclosed mixing well. Using a bristle brush applicator, apply 1 coat of SUPERB Universal Adhesive to the entire cavity preparation and the surrounding tooth structure, allow material to dwell for twenty seconds. DO NOT DRY.
4. Using the remaining material in the well and same applicator, apply 2 additional coats of SUPERB Universal Adhesive.
5. Dry the surface thoroughly. Do not blast the surface with air or use suction as this could remove material from the surface and lead to debonds or post operative sensitivity.
6. Light-activate for ten seconds. SUPERB Universal Adhesive can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

**Storage:**

Protect from sunlight. Protect from freezing. Store between 2°C/36°F and 30°C/86°F. See individual component labels for specific expiration dates.

**Cautions:**

Avoid contact with eyes. If the material come into contact with the eyes, flush with copious amounts of water and seek medical attention.

Un-polymerized resins may cause skin sensitization in susceptible persons, in case of contact with the skin, wash thoroughly with soap and water.

**Notes:**

Visit [vistaapex.com](http://vistaapex.com) for SDS

**Notes:**

1. Although it is not critical to the success of SUPERB Universal Adhesive, it is recommended that the preparation be cleaned with Clean and Boost® (Gel or Liquid) prior to beginning the bonding protocol. Pumicing or gentle particle abrasion of enamel is also recommended.
2. Deciduous teeth are treated in the same manner as permanent teeth.
3. Uncut enamel should be roughened or etched prior to bonding.
4. SUPERB Universal Adhesive is always light-activated before placing veneers and bonded crowns.