
VISTA APEX

 **SUPERB**
UNIVERSAL ADHESIVE

REF **SDL17, SDL18**

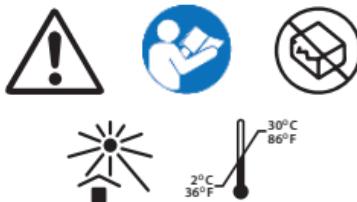Rx Only

Emergo Europe

Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Inter-Med, Inc.

2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755
www.vistaapex.com


90384-I-AP-MUL (2)

Catalogue Number

EN	Catalogue number
FR	Référence catalogue
DE	Bestellnummer
IT	Numero di catalogo
ES	Número de catálogo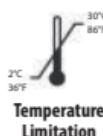

EN	Temperature limitation
FR	Limite de température
DE	Temperaturbegrenzung
IT	Limiti temperatura
ES	Límites de temperatura

**Caution,
Consult
Accompanying
Documents**

EN	Caution, consult accompanying documents
FR	Attention, consulter les documents joints
DE	Vorsicht, Begleitdokumente beachten
IT	Attenzione, consultare la documentazione allegata
ES	Precavión: consulte documentos adjuntos

EC REP

Authorized Representative in the European Community

EN	Authorized Representative in the European Community
FR	Représentant agréé pour la Communauté européenne
DE	Autorisierte Vertretung in der Europäischen Gemeinschaft
IT	Rappresentante autorizzato nella Comunità Europea
ES	Representante autorizado en la Comunidad Europea

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EN**Description:**

SUPERB Universal Adhesive is a single bottle bonding system for use in ALL dental restorations. SUPERB Universal Adhesive combines the etchant, primer and bonding resin into a single component. Through the elimination of components and steps within the application protocol, SUPERB Universal Adhesive is extremely easy to apply and has little to no technique sensitivity. SUPERB Universal Adhesive provides: easy application, consistent results, best in class bonding and the elimination of post operative sensitivity in a single bottle. SUPERB Universal Adhesive can be used in 3 modes: Self Etch, Selective Etch or Total Etch. All 3 techniques will provide exceptional results.

Indications for Use:

- Direct Restorations: to seal enamel / dentin prior to restoring with light-cure or self-cure composite material
- Indirect Restorations: bonding light-cured, self-cured or dual-cured composite cements or glass ionomer or resin-modified glass ionomer cements
- Bonding CEREC® restorations
- Desensitization: to treat hypersensitive and/or exposed root surfaces
- Desensitizing crown preparations
- Sealing preparations before placing amalgams
- Bonding crowns and bridges
- Bonding in posts and cores
- Bonding veneers

General Instructions^{1,2,3,4}:**MODE: SELF ETCH**

1. Rinse the preparation thoroughly and remove any pooled water. Leave tooth moist
2. Dispense a drop of SUPERB Universal Adhesive into the cavity of the enclosed mixing well. Using a bristle brush applicator, apply 1 coat of SUPERB Universal Adhesive to

the entire cavity preparation and the surrounding tooth structure, allow material to dwell for twenty seconds. DO NOT DRY.

3. Using the remaining material in the well and same applicator, apply 2 additional coats of SUPERB Universal Adhesive
4. Dry the surface thoroughly. Do not blast the surface with air or use suction as this could remove material from the surface and lead to debonds or post operative sensitivity.
5. Light-activate for ten seconds. SUPERB Universal Adhesive can be activated with any dental activation light.
6. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

MODE: SELECTIVE ETCH

Accidental etching of dentin will not negatively impact bond strengths.

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air. Do not desiccate the tooth surface.
2. Apply Clean & Boost® (Gel or Liquid) to the enamel surfaces to be bonded. Agitate for ten seconds, rinse thoroughly and leave moist.
3. Dispense a drop of SUPERB Universal Adhesive into the cavity of the enclosed mixing well. Using a bristle brush applicator, apply 1 coat of SUPERB Universal Adhesive to the entire cavity preparation and the surrounding tooth structure, allow material to dwell for twenty seconds. DO NOT DRY.
4. Using the remaining material in the well and same applicator, apply 2 additional coats of SUPERB Universal Adhesive.

5. Dry the surface thoroughly. Do not blast the surface with air or use suction as this could remove material from the surface and lead to debonds or post operative sensitivity.
6. Light-activate for ten seconds. SUPERB Universal Adhesive can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

MODE: TOTAL ETCH

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air. Do not desiccate the tooth surface.
2. Apply Clean & Boost® (Gel or Liquid) to the tooth surface to be bonded (both dentin and enamel). Agitate the material for 10 seconds , rinse thoroughly and leave moist.
3. Dispense a drop of SUPERB Universal Adhesive into the cavity of the enclosed mixing well. Using a bristle brush applicator, apply 1 coat of SUPERB Universal Adhesive to the entire cavity preparation and the surrounding tooth structure, allow material to dwell for twenty seconds. DO NOT DRY.
4. Using the remaining material in the well and same applicator, apply 2 additional coats of SUPERB Universal Adhesive.
5. Dry the surface thoroughly. Do not blast the surface with air or use suction as this could remove material from the surface and lead to debonds or post operative sensitivity.
6. Light-activate for ten seconds. SUPERB Universal Adhesive can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

Storage:

Protect from sunlight. Store between 2°C/36°F and 30°C/86°F. See individual component labels for specific expiration dates.

Cautions:

Avoid contact with eyes. If the material come into contact with the eyes, flush with copious amounts of water and seek medical attention. Un-polymerized resins may cause skin sensitization in susceptible persons, in case of contact with the skin, wash thoroughly with soap and water.

Notes:

Visit vistaapex.com for SDS

¹ Although it is not critical to the success of SUPERB Universal Adhesive, it is recommended that the preparation be cleaned with Clean and Boost® (Gel or Liquid) prior to beginning the bonding protocol. Pumicing or gentle particle abrasion of enamel is also recommended.

² Deciduous teeth are treated in the same manner as permanent teeth.

³ Uncut enamel should be roughened or etched prior to bonding.

⁴ SUPERB Universal Adhesive is always light-activated before placing veneers and bonded crowns.

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