VISTA APEX

REF \*90000, \*90001,  
90002, 90003, 90004

**Rx Only**

\* Asterisk notes CE marked SKU

CE  
0459

**Emergo Europe**

Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

**Inter-Med, Inc.**

2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755

[www.vistaapex.com](http://www.vistaapex.com)

90387-I-AP-MUL (2)

**Catalogue  
Number**

- EN Catalogue number  
FR Référence catalogue  
DE Bestellnummer  
IT Numero di catalogo  
ES Número de catálogo

**Caution,  
Consult  
Accompanying  
Documents**

- EN Caution, consult accompanying documents  
FR Attention, consulter les documents joints  
DE Vorsicht, Begleiddokumente beachten  
IT Attenzione, consultare la documentazione allegata  
ES Precaución: consulte documentos adjuntos

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**Authorized  
Representative  
in the European  
Community**

- EN Authorized Representative in the European Community  
FR Représentant agréé pour la Communauté européenne  
DE Autorisierte Vertretung in der Europäischen Gemeinschaft  
IT Rappresentante autorizzato nella Comunità Europea  
ES Representante autorizado en la Comunidad Europea

**Manufacturer**

- EN Manufacturer  
FR Fabricant  
DE Hersteller  
IT Produttore  
ES Fabricante

## Rx Only

- EN CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.  
FR ATTENTION: la loi fédérale américaine stipule que la vente de ce dispositif est réservée aux médecins ou sur prescription médicale.  
DE VORSICHT: US-Bundesgesetz beschränkt den Verkauf dieses Produkts an einen Arzt oder auf dessen Anordnung.  
IT ATTENZIONE: la legge federale statunitense limita la vendita di questo dispositivo a medici o su prescrizione medica.  
ES PRECAUCIÓN: las leyes federales de los Estados Unidos solo permiten la venta de este dispositivo por parte un médico o por prescripción facultativa.

**EN****Description:**

SURPASS® is a universal adhesive system that when properly employed, can be used to seal enamel/dentin prior to restoring with light-cured or self-cured composite materials. Also, in situations of indirect restorations, the bonding system can be used to seal a preparation when using a light-cured, self-cured or dual-cured composite cement or glass ionomer or resin-modified glass ionomer cement. In addition, the system allows the dentist to bond a post and core, along with treating hypersensitive and/or exposed root surfaces.

**Indications for Use:**

- Restorations: to seal enamel/dentin prior to restoring with light-cured or self-cured composite materials.
- Indirect Restorations: to seal a preparation when using a light-cured, self-cured or dual-cured composite cement or glass ionomer or resin-modified glass ionomer cement.
- Desensitization: to treat hypersensitive and/or exposed root surfaces.
- To bond in a post and core.

**General Instructions<sup>1,2,3,4</sup>:**

Although it is not critical to the success of SURPASS®, it is recommended that the preparation be cleaned with isopropyl alcohol on a cotton pellet and rinsed with water prior to beginning the bonding protocol. Pumicing or gentle particle abrasion of enamel is also recommended.

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air.
2. Dispense a drop of SURPASS® 1 into one cavity of the enclosed mixing well. Using the proper color-coded brush, apply SURPASS® 1 to the entire cavity preparation and the surrounding tooth structure, agitating briskly for at least ten seconds. **DO NOT DRY SURPASS® 1.**
3. Dispense a drop of SURPASS® 2 into another cavity of the mixing well.

Using the proper color-coded brush apply three brushfuls of SURPASS® 2 to the entire cavity preparation, dabbing each brushful to help distribute as much of the primer as possible.

4. Dry the primed surface with dried compressed air for at least ten seconds with the air syringes tip as close as possible to the surface. **DRY SURPASS® 2 VERY THOROUGHLY.**
5. Dispense a drop of SURPASS® 3 into another cavity of the mixing well. Using the proper color-coded brush apply a thin layer of SURPASS® 3 to the entire cavity surface.
6. Light-activate for ten seconds. SURPASS® can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

**Indirect Restorations<sup>1,2,3,4</sup>:**

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air.
2. Dispense a drop of SURPASS® 1 into one cavity of the enclosed mixing well. Using the proper color-coded brush, apply SURPASS® 1 to the entire cavity preparation and the surrounding tooth structure, agitating briskly for at least 10 seconds. **DO NOT DRY SURPASS® 1.**
3. Dispense a drop of SURPASS® 2 into another cavity of the mixing well. Using the proper color-coded brush apply three brushfuls of SURPASS® 2 to the entire cavity preparation, dabbing each brushful to help distribute as much of the primer as possible.
4. Dry the primed surface with dried compressed air for at least 10 seconds with the air syringe tip as close as possible to the surface. **DRY SURPASS® 2 THOROUGHLY.**
5. Dispense a drop of SURPASS® 3 into another cavity of the mixing well. Using the proper color-coded brush apply a thin layer of SURPASS® 3 to the entire

cavity surface. Air-thin the SURPASS®3 aggressively. Final file thickness will be around 10 – 12 microns.

6. Light-activate for 10 seconds. SURPASS® can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

#### **Metal Bonding<sup>1,2,3,4:</sup>**

1. Roughen the metal surface to be bonded. Ideal the surface should be micro-etched with 27 micron aluminum oxide powder. If micro-etching is not available the surface should be roughened with a burr.
2. Rinse and dry the surface.
3. Dispense a drop of SURPASS®2 into a cavity of the mixing well. Using the proper color-coded brush apply two brushfuls of SURPASS®2 to the metal surface.
4. Dry the primed surface with dried compressed air for at least 10 seconds with the air syringe tip as close as possible to the surface.  
**DRY SURPASS®2 THOROUGHLY.**
5. Dispense a drop of SURPASS®3 into another cavity of the mixing well. Using the proper color-coded brush apply a thin layer of SURPASS®3 to the entire cavity surface.
6. Light-activate for 10 seconds. SURPASS® can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

#### **Bonding to Existing Composite<sup>1,2,3,4:</sup>**

1. Roughen the composite surface to be bonded. Ideal the surface should be micro-etched with 27 micron aluminum oxide powder. If micro-etching is not available the surface should be roughened with a burr.
2. Rinse and dry the surface.
3. Dispense a drop of SURPASS®2 into a cavity of the mixing well.

Using the proper color-coded brush apply two brushfuls of SURPASS®2 to the metal surface.

4. Dry the primed surface with dried compressed air for at least 10 seconds with the air syringe tip as close as possible to the surface.  
**DRY SURPASS®2 THOROUGHLY.**
5. Dispense a drop of SURPASS®3 into another cavity of the mixing well. Using the proper color-coded brush apply a thin layer of SURPASS®3 to the entire cavity surface.
6. Light-activate for 10 seconds. SURPASS® can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

#### **Storage:**

Protect from sunlight. Store between 2°C/36°F and 30°C/86°F. This product is subject to evaporation, keep bottles tightly capped. See individual component labels for specific expiration dates.

#### **Cautions:**

Avoid contact with eyes. If the material comes into contact with the eyes, flush with copious amounts of water and seek medical attention. Un-polymerized resins may cause skin sensitization in susceptible persons, in case of contact with the skin, wash thoroughly with soap and water.

#### **Notes:**

Visit [vistaapex.com](http://vistaapex.com) for CAD/CAM Restoration Instructions, Sealing and Imaging for a CEREC® Restoration, and for SDS.

<sup>1</sup> For indirect restorations, air-thin SURPASS®3 sufficiently to allow seating of the restoration and light-activate.

<sup>2</sup> Deciduous teeth are treated in the same manner as permanent teeth.

<sup>3</sup> SURPASS®1 should be agitated for 15 seconds on uncut enamel.

<sup>4</sup> SURPASS® is always light-cured before placing veneers and bonded crowns.

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