

# TROUBLESHOOTING GUIDE: SURPASS® SELF-ETCH ADHESIVE SYSTEM

## ***Sensitivity:***

- Leave SURPASS® 1 wet. Do not air thin SURPASS® 1.
- After applying SURPASS® 2, dry thoroughly.

## ***Poor bonding:***

- After applying SURPASS® 2, dry thoroughly.
- Do not pretreat the tooth with desensitizers.
- Always use a fresh applicator tip to apply SURPASS® 2.
- Do not apply SURPASS® 2 with the tip that was used to apply SURPASS® 1.
- For core applications, light-cure the first 1mm layer of D/C cement prior to bulk filling the core.

Please visit our website at [www.vistaapex.com](http://www.vistaapex.com) to view the full instructions for use.



*Scan to learn more  
about the SURPASS®  
Self-Etch Adhesive System*



VISTA APEX  
[vistaapex.com](http://vistaapex.com)

## Directions:

1. Apply SURPASS® 1 and agitate for 10 seconds.  
**LEAVE WET.**
2. **USING A CLEAN APPLICATOR**, apply 3 coats of SURPASS® 2.
3. **DRY THOROUGHLY.**
4. Apply a thin layer of SURPASS® 3.
5. Light cure for 10 seconds.


Please visit our website at [www.vistaapex.com](http://www.vistaapex.com) to view the full instructions for use.




VISTA APEX

[vistaapex.com](http://vistaapex.com)


# SURPASS® TECHNIQUE CARD

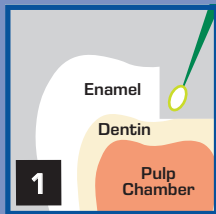


Apply SURPASS® 1 and agitate for 10 sec.  
**LEAVE WET.**

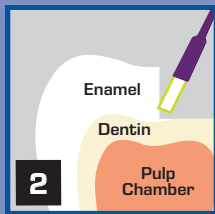


Apply 3 coats of SURPASS® 2.

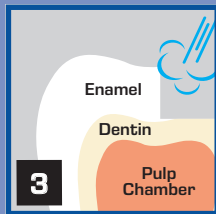

**DRY THOROUGHLY.**

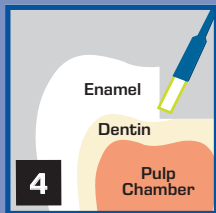

Apply a thin layer of SURPASS® 3.

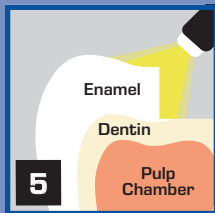

Light cure for 10 sec.

### Notes:

- For indirect restorations, air-thin SURPASS® 3 sufficiently to allow seating of the restoration and light-activate.
- Deciduous teeth are treated in the same manner as permanent teeth.
- SURPASS® 1 should be agitated for 15 seconds on uncut enamel.
- SURPASS® is always light-activated before placing veneers and bonded crowns.