VISTA | APEX

REF 90413

Rx Only

CE  
0459

**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755

90546-I-AP (1)

**Do not use if package is damaged**

EN	Do not use if package is damaged
BG	Не използвайте, ако опаковката е повредена
CZ	Nepoužívejte, pokud je obal poškozen
DA	Må ikke anvendes, hvis pakningen er beskadiget
ET	Ärge kasutage, kui pakend on kahjustatud
FR	Ne pas utiliser si l'emballage est endommagé
DE	Nicht verwenden, wenn die Verpackung beschädigt ist
EL	Μην το χρησιμοποιείτε εάν η συσκευασία είναι κατεστραμμένη
HU	Ne használja, ha a csomagolás sérült
IT	Non utilizzare se la confezione è danneggiata
LV	Nelietot, ja iepakojums ir bojāts
LT	Nenaudokite, jei pakuotė pažeista
PT	Não use se o pacote estiver danificado
RO	Nu utilizați dacă ambalajul este deteriorat
SK	Nepoužíajte, ak je obal poškodený
ES	No utilice el producto si el envoltorio está dañado
SV	Får inte användas om förpackningen är skadad

**Caution, consult accompanying documents**

EN	Caution, consult accompanying documents
BG	Внимание, вижте придружаващите документи
CZ	Pozor, prostudujte si přírodní dokumenty
DA	Forsigtig, læs de medfølgende dokumenter
ET	Ettevaatus, tutvuge kaasasolevate dokumentidega
FR	Attention, consultez les documents joints
DE	Vorsicht, Begleitdokumente beachten
EL	Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα
HU	Vigyázat, olvassa el a kísérelő dokumentumokat
IT	Attenzione, consultare la documentazione allegata
LV	Uzmanību, skatiet pavaddokumentus
LT	Atsargiai, skaitykite pridedamus dokumentus
PT	Cuidado, consulte os documentos anexos
RO	Atenție, consultați documentele însoțitoare
SK	Pozor, pozrite si sprievodní dokumentáciu
ES	Precaución: consulte documentos adjuntos
SV	Lakttag försiktighet, konsultera medföljande dokument

**Caution, consult accompanying documents**

EN	Temperature limitation
BG	Ограничение на температурата
CZ	Omezení teploty
DA	Temperaturbegrænsning
ET	Temperatuuripiiramine
FR	Limite de température
DE	Temperaturbegrenzung
EL	Περιορισμός θερμοκρασίας
HU	Hőmérséklet korlátozás
IT	Limiti temperatura
LV	Temperatūras ierobežojums
LT	Temperatūros apribojimas
PT	Limitação de temperatura
RO	Limitarea temperaturii
SK	Obmedzenie teploty
ES	Limites de temperatura
SV	Temperaturgräns

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**EN**

DesensiMAX is a powerful desensitizer that acts to prevent bond degradation, rewets the tooth surface prior to bonding, and optimizes the tooth / restoration interface. Bonding protocols often call for a "moist" or "shiny" or "damp" surface for bonding. DesensiMAX eliminates the guess work by rewetting to the proper level every time. Finally, DesensiMAX provides fluoride and proprietary ingredient to the surface of the tooth to create a zone of inhibition and prevent bond degradation caused by the formation of matrix metalloproteinase (MMP's).

**Indications for Use:**

- Desensitizing
- Rewetting tooth surface prior to bonding
- Preventing bond degradation

**General Instructions**

**NOTE: ALWAYS SHAKE BOTTLE VIGOROUSLY FOR 3 SECONDS BEFORE DISPENSING**

**MODE: TOTAL ETCH**

1. Etch the tooth surface as instructed by the adhesive manufacturer
2. Rinse away the extra etch and dry the tooth thoroughly for at least 3 seconds
3. Using a small micro-applicator, apply a uniform coat of DesensiMAX to the entire preparation. Allow to dwell for 5 seconds, dry briefly, leaving a shiny surface. Surface should not have any pooled liquid
4. Continue to apply the adhesive manufacturer's materials as prescribed in their instructions

**MODE: SELECTIVE ETCH**

1. Etch the enamel surface as instructed by the adhesive manufacturer
2. Rinse away the extra etch and dry the tooth thoroughly for at least 3 seconds
3. Using a small micro-applicator, apply a uniform coat of DesensiMAX to the entire preparation. Allow to dwell for 5 seconds, dry briefly, leaving a shiny surface. Surface should not have any

pooled liquid.

4. Continue to apply the adhesive manufacturer's materials as prescribed in their instructions.

**DESENSITIZING CROWN PREPARATIONS:**

1. Prior to cementation of the temporary, dry the tooth aggressively for at least 3 seconds

**2. NOTE: SHAKE BOTTLE VIGOROUSLY FOR 3 SECONDS BEFORE DISPENSING**

3. Apply 2 coats of DesensiMAX, allow to dwell for 5 seconds, briefly dry and place the temporary

4. For maximum effect, repeat the application several times, i.e. aggressively dry the prepared tooth, apply 2 coats of DesensiMAX, allow to dwell for 5 seconds, dry again

5. The clinician may either choose to proceed to cement the temporary crown or apply either Surpass® or Superb to seal the crown preparation and then cement the temporary

**TREATING SENSITIVE DENTIN:**

1. Dry the dentin as much as possible
2. Apply a brushful of DesensiMAX
3. Allow to dwell 5 seconds
4. Dry
5. Apply another brushful of DesensiMAX
6. Dry
7. Repeat until sensitivity disappears
8. If desired, apply a coat of Surpass® 2 or Superb, dry, and light activate to prolong the desensitization

**NOTE:** If using with a single bottle adhesive in strict self etch mode (no etching) or with a resin modied glass ionomer, the BondSaver product should be used in place of DesensiMAX.

**Precautions:**

1. Do not breathe dust/fume/gas/mist/vapors/spray
2. Wash hands thoroughly after handling
3. Wear protective gloves, eye protection
4. Store in a well-ventilated area
5. Keep cool, do NOT store in direct sunlight

**Storage:**

Protect from sunlight. Store between 2°C/36°F and 30°C/86°F. See individual component labels for specific expiration dates.

NOTE: See [vistaapex.com](http://vistaapex.com) for SDS

**WARNING:**

Causes serious eye irritation.

Wear eye/face protection. Prolonged or repeated skin contact may cause severe irritation.

**FIRST AID:**

IF IN EYES: Rinse cautiously with water for several minutes, also under eye lids. Remove contact lenses, if present and easy to do so.

IF INHALED: Move to fresh air and keep at a rest position comfortable for breathing.

IF INGESTED: Do not induce vomiting. Rinse mouth, drink plenty of water. Call a physician or Poison Control Center immediately

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