

## Troubleshooting Guide


### Sensitivity:

- Leave BOTTLE 1 wet. Do not dry or air thin BOTTLE 1.
- After applying BOTTLE 2, dry thoroughly.

### Poor bonding:

- After applying BOTTLE 2, dry thoroughly.
- Do not pretreat the tooth with desensitizers.
- Always use a fresh applicator tip to apply BOTTLE 2.
- Do not apply BOTTLE 2 with the tip that was used to apply BOTTLE 1.
- For core applications, light-cure the first 1mm layer of D/C cement prior to bulk filling the core.

Please visit our website at [www.vistaapex.com](http://www.vistaapex.com) to view the full instructions for use.

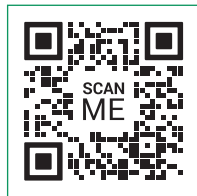

*Scan to learn more*



VISTA APEX

**[vistaapex.com](http://vistaapex.com)**

## Directions:

1. Apply BOTTLE 1 and agitate for 10 seconds.  
**LEAVE WET.**
2. **SHAKE BOTTLE 2** prior to dispensing. **USING A CLEAN APPLICATOR**, apply 3 coats of BOTTLE 2.
3. **DRY THOROUGHLY.**
4. Apply a thin layer of BOTTLE 3.
5. Light-cure 10 seconds.


Please visit our website at  
[www.vistaapex.com](http://www.vistaapex.com) to view  
the full instructions for use.

**VA** VISTA APEX


[vistaapex.com](http://vistaapex.com)

# RE-GEN™


## BIOACTIVE SELF-ETCH ADHESIVE TECHNIQUE CARD

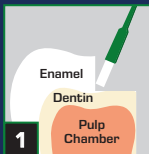


Apply BOTTLE 1 and agitate for 10 sec. **LEAVE WET.**

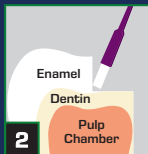


**SHAKE BOTTLE 2** prior to dispensing. **USING A CLEAN APPLICATOR**, apply 3 coats of BOTTLE 2.

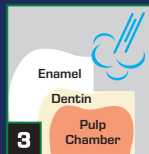

**DRY THOROUGHLY.**

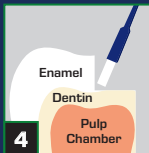

Apply a thin layer of BOTTLE 3.

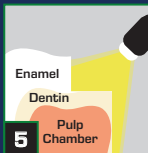

Light-cure 10 seconds.

### Notes:

- For indirect restoration, air-thin BOTTLE 3 sufficiently to allow seating of the restoration and light-activate.
- Deciduous teeth are treated in the same manner as permanent teeth.
- BOTTLE 1 should be agitated for 15 seconds on uncut enamel.

*RE-GEN™ Bioactive Self-Etch Adhesive is always light-activated before placing veneers and bonded crowns.*