VISTA APEX

REGEN™
BIOACTIVE UNIVERSAL ADHESIVE

REF *90637
*90638
90706

Rx Only

*Only asterisk SKUs are CE Marked

CE
0459

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Inter-Med, Inc.
2200 South St.
Racine, WI 53404, USA
Tel: +1-262-636-9755

91026-I-AP-MUL (3)

Do not use
if package is
damaged

EN	Do not use if package is damaged
BG	Не използвайте, ако опаковката е повредена
CZ	Nepoužívejte, pokud je obal poškozen
DA	Må ikke anvendes, hvis pakningen er beskadiget
ET	Ärge kasutage, kui pakend on kahjustatud
FR	Ne pas utiliser si l'emballage est endommagé
DE	Nicht verwenden, wenn die Verpackung beschädigt ist
EL	Μην το χρησιμοποιείτε εάν η συσκευασία είναι κατεστραμμένη
HU	Ne használja, ha a csomagolás sérült
IT	Non utilizzare se la confezione è danneggiata
LV	Nelietot, ja iepakojums ir bojāts
LT	Nenaudokite, jei pakuotė pažeista
PT	Não use se o pacote estiver danificado
RO	Nu utilizați dacă ambalajul este deteriorat
SK	Nepoužívajte, ak je obal poškodený
ES	No utilice el producto si el envoltorio está dañado
SV	Får inte användas om förpackningen är skadad

Caution, consult
accompanying
documents

EN	Caution, consult accompanying documents
BG	Внимание, виекте придружаващите документи
CZ	Pozor, prostudujte si přírodní dokumenty
DA	Forsigtig, læs de medfølgende dokumenter
ET	Ettevaatus, tutvuge kaasasolevate dokumentidega
FR	Attention, consulter les documents joints
DE	Vorsicht, Begleitdokumente beachten
EL	Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα
HU	Vigyázat, olvassa el a kísérelő dokumentumokat
IT	Attenzione, consultare la documentazione allegata
LV	Uzmanību, skatiet pavaddokumentus
LT	Atsargiai, skaitykite pridedamus dokumentus
PT	Cuidado, consulte os documentos anexos
RO	Atenție, consultați documentele însoțitoare
SK	Pozor, pozrite si sprievodní dokumentáciu
ES	Precaución: consulte documentos adjuntos
SV	Lakttag försiktighet, konsultera medföljande dokument

Caution, consult
accompanying
documents

EN	Temperature limitation
BG	Ограничение на температурата
CZ	Omezení teploty
DA	Temperaturbegrænsning
ET	Temperatuuri piiramine
FR	Limite de température
DE	Temperaturbegrenzung
EL	Περιορισμός θερμοκρασίας
HU	Hőmérséklet korlátozás
IT	Limiti temperatura
LV	Temperatūras ierobežojums
LT	Temperatūros apribojimas
PT	Limitação de temperatura
RO	Limitarea temperaturii
SK	Obmedzenie teploty
ES	Limites de temperatura
SV	Temperaturgräns

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EN**Description:**

RE-GEN™ Bioactive Universal Adhesive System is a single bottle, light-cure only, universal bonding system that delivers calcium, phosphate and fluoride ions directly to compromised tooth structures to promote regeneration. RE-GEN™ Bioactive Universal Adhesive can be used in all restorations, both direct and indirect, and offers the flexibility for self-etch, selective etch or total etch procedures.

Indications for Use:

- Direct Restorations: to seal enamel / dentin prior to restoring with light-cure or self-cure composite material.
- Indirect Restorations: bonding light-cured, self-cured or dual-cured composite cements or glass ionomer or resin-modified glass ionomer cements
- Bonding CEREC® restorations
- Desensitization: to treat hypersensitive and / or exposed root surfaces
- Desensitizing crown preparations
- Sealing preparations before placing amalgams
- Bonding crowns and bridges
- Bonding in posts and cores
- Bonding veneers

General Instructions ^{1,2,3,4}:

1. Rinse the preparation thoroughly and remove all visible water by drying the surface for approximately 5 seconds with dried compressed air. Do not desiccate the tooth surface.
2. Apply Clean & Boost® to the tooth surface to be bonded (both dentin and enamel) and agitate for 10 seconds, rinse thoroughly and leave moist. Clean & Boost® is critical to the procedure to remove any contaminants that could impede the ions from penetrating the tooth structure.
3. Dispense a drop of RE-GEN™ Bioactive Universal Adhesive into the cavity of the enclosed mixing well. Using an applicator bristle-type brush, apply 1 coat of RE-GEN™ Bioactive Universal Adhesive to the entire cavity preparation and the surrounding tooth structure, allow to dwell for 20 seconds. **DO NOT DRY.**

4. Using the remaining material in the well and same applicator, apply 2 additional coats of RE-GEN™ Bioactive Universal Adhesive.
5. Dry the surface thoroughly. Do not blast the surface with air or use suction as this could remove material from the surface and lead to debonds or post-operative sensitivity.
6. Light-activate for ten seconds. RE-GEN™ Bioactive Universal Adhesive can be activated with any dental activation light.
7. Continue restorative procedure with material of choice. Follow manufacturer's instructions for all restorative materials.

Storage:

Protect from sunlight. Store between 2°C/36°F and 30°C/86°F. See individual component labels for specific expiration dates.

Cautions:

Avoid contact with eyes. If the material comes into contact with the eyes, flush with copious amounts of water and seek medical attention. Un-polymerized resins may cause skin sensitization in susceptible persons, in case of contact with the skin, wash thoroughly with soap and water.

Notes:

1. Deciduous teeth are treated in the same manner as permanent teeth
2. Uncut enamel should be roughened or etched prior to bonding
3. RE-GEN™ Bioactive Universal Adhesive is always light-activated before placing veneers and bonded crowns
4. Visit vistaapex.com for SDS

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