



Indications:

Vista-Cal™ Calcium Hydroxide paste is a 12.5pH aqueous solution containing no more than 35% calcium hydroxide. The radiopaque solution may be used in conventional endodontic treatments, pulp capping, fracture repair or as a temporary intra-canal dressing.

Instructions for Use:

1. Isolate area to be treated with a rubber dam.
2. Remove Vista-Cal™ syringe and applicator tip from their packaging.
3. Remove syringe cap and attach tip securely to syringe. Apply as desired: only small amounts are necessary for pulp capping or canal dressing. During apexification retreatment from the canal. (The use of a plugger may assist in packing the material).
5. Remove and dispose of tip from the syringe
6. Clean and disinfect syringe between patients

STORE AT ROOM TEMPERATURE



WARNING

HARMFUL IF SWALLOWED. KEEP OUT OF REACH OF CHILDREN. AVOID CONTACT WITH SKIN AND EYES. MAY BE HARMFUL IF SWALLOWED CAUSE MILD SKIN IRRITATION CAUSES EYE IRRITATION. WASH HANDS THOROUGHLY AFTER HANDLING. WEAR PROTECTIVE GLOVES/EYE PROTECTION. DISPOSE OF CONTENTS IN ACCORDANCE WITH LOCAL, STATE, AND FEDERAL REGULATIONS.



FIRST AID: IF SWALLOWED: CALL A DOCTOR IF YOU FEEL UNWELL. RINSE MOUTH. **IF IN EYES:** RINSE CAUTIOUSLY WITH WATER FOR SEVERAL MINUTES. REMOVE CONTACT LENSES, IF PRESENT AND EASY TO DO SO. CONTINUE RINSING. **IF ON SKIN:** WASH WITH PLENTY OF SOAP AND WATER.



Inter-Med, Inc.
2200 South St.,
Racine, WI 53404 USA
Tel: +1-262-636-9755

Rx ONLY

502600-I-V-ENG (7)



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands
Tel: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299



0459

Made in USA