



VISTA | APEX

Chlor-XTRA™ (Patented)

REF 503825 - 3cc Pre-filled Syringes

REF 503850 - 12cc Pre-filled Syringes



Description:

Chlor-XTRA™ is a 6% sodium hypochlorite formula; enhanced with powerful wetting agents and proprietary surface modifiers that penetrate into lateral canals and isthmuses by lowering the surface tension, yielding a much more powerful oxidizing agent, 2+ times thinner than standard sodium hypochlorite and 2.6 times more digestive. It is used for the irrigation and debridement of root canals during and after instrumentation.

Instructions for Use:

1. Remove syringe cap and attach irrigating tip securely to syringe.
2. Irrigate with Chlor-XTRA™ as needed for debridement during instrumentation.
3. Remove smear layer with appropriate irrigant, such as EDTA solution, Citric Acid 12.5% or SmearOFF™ 2-in-1.
4. Irrigate again with Chlor-XTRA™ for further debridement.
5. Dispose of syringe and irrigating tip after use.

Precautions:

1. Read SDS for material handling and safety information.
2. Avoid allowing Chlor-XTRA™ to be ingested, use proper evacuation.
3. Avoid spilling on colored fabrics and surfaces.

Storage:

- Store under refrigeration.
- Bring solution to room temperature prior to use.



DANGER

Skin corrosion. Serious eye damage. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/eye protection/face protection/protective clothing. Harmful if swallowed. Wash thoroughly after handling to avoid any chemical contact with eyes, skin and clothing.



Keep container tightly closed to avoid inhalation or accidental ingestion. Use adequate ventilation when necessary. Store in a tightly closed container. Follow all government regulations for waste disposal. Do not flush product into sewer or drains that may lead to waterways. **FIRST AID:**

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** take off contaminated clothing and shoes immediately. Wash off with soap and plenty of water. **IF INHALED:** Move person to fresh air. **IF SWALLOWED:**

Do not induce vomiting. Rinse mouth with water. Specific Treatment: Contact a POISON CONTROL CENTER or doctor/physician. **FIRE: IN CASE OF FIRE:**

Evacuate all personnel. Use extinguishing media most appropriate for the surrounding fire. Substance is non combustible. Nonflammable; contact with metals may evolve flammable hydrogen gas.



Inter-Med, Inc.
2200 South St.,
Racine, WI 53404 USA
Tel: +1-262-636-9755

Rx ONLY
Made in USA
503800-I-V-ENG (5)



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands
Tel: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299