



REF 317002

REF 503700

Indications:

EDTA solution (17%); necessary for the removal of the smear layer and dentin mud.

Instructions for Use:

1. Remove EDTA syringe.
2. Remove syringe cap and attach irrigation tip securely to syringe.
3. Irrigate as desired.
4. Dispose of irrigating tip and syringe.

Warnings:

**** NOT TO BE USED AS AN ORAL RINSE ****

- Always replace cap after use.
- Avoid contact with skin and eyes.

**** STORE AT ROOM TEMPERATURE ****



WARNING

May be harmful if swallowed.
Causes mild skin irritation.
Causes eye irritation.

Wash hands thoroughly after handling. Wear protective gloves/eye glasses. Follow all government regulations for waste disposal. Prevent release to the environment if possible. Do not flush waste into sewer or waterways.

FIRST AID: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water.



Inter-Med, Inc.
2200 South St.,
Racine, WI 53404, USA
Tel: +1-262-636-9755

Made in USA

Rx ONLY

317002-I-V-ENG (5)



CE
0459

Emergo Europe

Prinsessegracht 20
2514 AP The Hague
The Netherlands

Tel: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299