

Appli-Vac™ Aspirators

Endodontic Root Canal/Tip

Note:

- Follow manufacturers instructions and guidelines for maintenance of endodontic root canal/tip aspirators.

Daily Maintenance:

- Flush aspirator thoroughly with water between each patient to reduce potential of biofilm build up.
- Insert and remove the supplied cleaning stylet multiple times to assist in removing trapped particles.
- Flush aspirator thoroughly with water.

Tip Management:

- Begin by straightening the tubing by pinching the tube at the base and pulling out to the tip.
- To re-curve aspirator tip, pinch the tubing at the base and re-curve accordingly.
- To prevent crimping, avoid drastic bends.

CAUTION:

- DO NOT BEND the 0.5mm root canal aspirator, Part No. 804001 (equivalent to item 01BA).