



Indications:

Vista Porcelain Etch kit is to be used any time resin bonding to porcelain is required. Porcelain Etch is used for the preparation of porcelain crowns, veneers or inlays prior to cementation. It is also used in preparing fractured porcelain and porcelain-to-metal crown bonding.

Instructions for use:

1. Isolate area to be etched with rubber dam.
2. Remove porcelain syringe and Spira-Flo™ Brush tips from their packaging.
3. Remove syringe cap and attach tip securely to syringe.
4. If preparing fractured porcelain for bonding, first smooth off the surface of the porcelain being etched with a medium grit bur, removing the glazed surface. Rinse away all particles with water and dry prepared porcelain with moisture/oil-free dry air.
5. Apply Vista Porcelain Etch with brush tip to the prepared porcelain surface.
6. Let stand for 60 seconds, then rinse with water and dry surface (completely removing etchant).
7. Etched porcelain should appear frosty white. If not, repeat steps 5 and 6.
8. Apply Best-Etch to remove salts and debris (5 seconds), rinse and dry.
9. Remove Silane syringe and Spira-Flo™ Brush tip from packaging.
10. Remove syringe cap and attach tip securely to syringe.
11. Brush Vista Silane onto etched porcelain surface and let dry completely. If needed, dry with moisture/oil-free air.
12. Place bonding resin according to manufacturer's instructions.



DANGER

May cause eye, respiratory and skin irritation. Harmful if swallowed. Wear gloves/eye protection. May be disposed of in landfill or incinerator in accordance with local, state and federal regulations. **FIRST AID: IF IN EYES:** Flush with large quantities of water for fifteen (15) minutes. Consult a physician. **IF ON SKIN:** Wash with soap and water.



IF SWALLOWED: Drink two glasses of milk or water. Seek medical attention if a substantial amount has been ingested. Do not induce vomiting.



Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755

Made in USA

Rx ONLY

502500-I-V-ENG (7)



Emergo Europe

Prinsessegracht 20
2514 AP The Hague
The Netherlands
Tel: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299