



REF 502925 - Endo Kit

REF 503400 - 30mL Bulk Syringe Kit

REF 503930 - Endo Refill

REF 503455 - Pre-filled Syringes Refill

REF 503650 - 30mL Bulk Syringe

REF 503925 - Unit Dose

Indications:

Used for the final irrigation step for long lasting cleansing.

Instructions for Use:

1. Remove CHX-Plus™ from syringe and irrigation tip from packaging.
2. Remove syringe cap and attach irrigation tip securely to syringe.
3. Irrigate as desired.
4. Remove and dispose of tip from the syringe.
5. Clean and disinfect syringe between patients.

Warnings:

**** NOT TO BE USED AS AN ORAL RINSE ****

- Irrigating tips are disposable.
- Always replace cap after use.
- Avoid contact with skin and eyes.



WARNING

May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation.

Wash hands thoroughly after handling. Wear protective gloves/eye glasses. Follow all government regulations for waste disposal. Prevent release to the environment if possible. Do not flush waste into sewer or waterways.

FIRST AID: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water.



MFD by: **Magnum
For: Inter-Med, Inc.**
2200 South St.
Racine, WI 53404 US
Tel: +1-262-636-9755
www.vistaapex.com

Rx ONLY

502455-I-EU-V-ENG (3)



Emergo Europe

Prinsessegracht 20
2514 AP The Hague
The Netherlands
Tel: +31(0)70 345 8570
Fax: +31(0)70 346 7299