

GETTING STARTED

Unpack the contents of the Phasor™ Kit.

Remove battery pull tab & connect the micro USB charger to the charging port.

The blue light will pulse when the battery is charging and will turn **solid blue** when the device is fully charged.

Battery charge is indicated through the **ON/OFF button**.



LOW BATTERY

INSERTING COMPOSITE

Snap a composite compule into the end of the device. Position the compule so that the tip is directly under the LED lens.

- **Short Compules:** position forward
- **Long Compules:** position back

NOTE: Phasor™ will not turn on unless composite has been inserted.

CAUTION: Parts of the composite compule may become hot. Use caution and avoid touching the top of a hot composite compule.



COMPOSITE & SETTING COMPATIBILITY

For a complete list of compatible compules and their ideal setting visit www.vista-dental.com/phasor

■ OPTIMAL Setting ■ Compatible Setting ■ DO NOT USE

COMPOSITE BRAND	SETTING 1	SETTING 2	SETTING 3	SETTING 4
3M**	Yellow	Yellow	Green	Red
DENTSPLY**	Yellow	Yellow	Green	Red
Ivoclar**	Yellow	Green	Green	Yellow
Kerr**	Yellow	Green	Red	Red
VOCO**	Yellow	Yellow	Yellow	Green
Coltene**	Yellow	Green	Red	Red
Kuraray**	Yellow	Green	Green	Yellow
GC**	Yellow	Green	Red	Red
Shofu**	Green	Red	Red	Red
Heraeus Kulzer**	Green	Red	Red	Red
Tokuyama**	Yellow	Green	Green	Red
Centrix®	Yellow	Yellow	Green	Red

CAUTION: Do not deviate from the recommended settings.



Refer to instruction manual for detailed instructions.

SELECTING A SETTING



Reference setting compatibility chart to determine the appropriate setting per composite brand.



CAUTION: Do not deviate from the recommended settings.



Press & hold to enter programming mode. This is indicated by flashing yellow lights.



Short press to toggle between settings.

1. ● ○ ○ ○ 3. ○ ○ ● ●
2. ○ ● ○ ○ 4. ● ● ● ●



Press & hold to exit. Device will beep 3 times and shut off.



Your setting is now memorized as default.

START HEATING CYCLE



Short press to start/stop heating cycle.



The blue lights will flash while heating.

Setting 1-3: 45 seconds

Setting 4: 70 seconds



The blue lights will turn solid when heating cycle is complete.



The device will automatically shut off after 3 minutes of sustained heating.

CLEANING & DISINFECTION

The device may be wiped down with a standard disinfectant wipe. Never spray disinfectant directly onto the device.



CAUTION: Always use a protective barrier.

Black nosecone is removable and autoclavable.

Between uses, check the lens to ensure it is clean. A dirty lens will decrease performance.



Refer to instruction manual for detailed instructions.