

GETTING STARTED

Unpack the contents of the Phasor™ Kit.

Remove battery pull tab & connect the micro USB charger to the charging port.

The blue light will pulse when the battery is charging and will turn **solid blue** when the device is fully charged.

Battery charge is indicated through the **ON/OFF button**.




INSERTING COMPOSITE


Snap a composite compule into the end of the device.
Position the compule so that the tip is directly under the LED lens.

- **Short Compules:** position forward
- **Long Compules:** position back

NOTE: Phasor™ will not turn on unless composite has been inserted.



CAUTION: Parts of the composite compule may become hot.
Use caution and avoid touching the top of a hot composite compule.

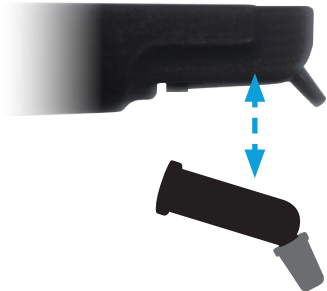

COMPOSITE & SETTING COMPATIBILITY

The chart below will list the ideal settings for most leading composite brands. Contact Vista Apex for additional assistance.

■ OPTIMAL Setting ■ Compatible Setting ■ DO NOT USE

COMPOSITE BRAND	SETTING 1	SETTING 2	SETTING 3	SETTING 4
3M**				
DENTSPLY**				
Ivoclar**				
Kerr**				
VOCO**				
Coltene**				
Kuraray**				
GC**				
Shofu*				
Heraeus Kulzer**				
Tokuyama**				
Centrix**				



CAUTION: Do not deviate from the recommended settings.

*3M® is a registered trademark of 3M Company.
Dentsply® is a registered trademark of DENTSPLY International.
Ivoclar® is a registered trademark of IVOCCLAR VIVADENT.
Kerr® is a registered trademark of Kerr Corporation.
VOCO® is a registered trademark of VOCO AMERICA, INC.
Coltene® is a registered trademark of COLTENE/WHALENENT, INC.
Kuraray® is a registered trademark of KURARAY CO., LTD.
GC® is a registered trademark of GC Corporation.
Shofu® is a registered trademark of SHOFU, INC.
Heraeus Kulzer® is a registered trademark of Heraeus Kulzer GmbH.
Tokuyama® is a registered trademark of Tokuyama Corporation.
Centrix® is a registered trademark of Centrix, Inc.

SELECTING A SETTING



Reference setting compatibility chart to determine the appropriate setting per composite brand.



CAUTION: Do not deviate from the recommended settings.



Press & hold to enter programming mode. This is indicated by flashing yellow lights.



Short press to toggle between settings.

1. 3.
2. 4.



Press & hold to exit. Device will beep 3 times and shut off.



Your setting is now memorized as default.

START HEATING CYCLE



Short press to start/stop heating cycle.



The blue lights will flash while heating.

Setting 1-3: 45 seconds **Setting 4:** 70 seconds



The blue lights will turn solid when heating cycle is complete.



The device will automatically shut off after 3 minutes of sustained heating.

CLEANING & DISINFECTION

The device may be wiped down with a standard disinfectant wipe. Never spray disinfectant directly onto the device.



CAUTION: Always use a protective barrier.

Black nosecone is removable and autoclavable.

Between uses, check the lens to ensure it is clean. A dirty lens will decrease performance.



Refer to User's Manual for detailed instructions.



408801-I-V-77-ENG (3)