



VISTA | APEX

**ZR-C<sup>TM</sup>**  
**UNIVERSAL CLEANSER**

**REF** 90782, 90779

Made in USA  
91165-I-V-ENG (2)

**Description:**

ZR-C™ is an aqueous-based universal cleanser suitable for non-abrasive cleaning of the intaglio surface of prosthetic restorations after intra-oral try-in.

**Indications for Use:**

Cleaning of ceramic and metal substrates prior to bonding or cementing in place.

**General Instructions (Syringe):**

1. After intra-oral try-in and determining the restoration is ready to deliver, thoroughly rinse with water to remove any contamination, and dry with oil-free air.
2. Attach a Spira-Flo™ Brush Tip to the ZR-C™ syringe, and dispense using a gentle brushing motion to cover the entire intaglio surface of the restoration with ZR-C™.
3. Allow to dwell for 20 seconds, then thoroughly rinse with water and dry with oil-free air.
4. Use Vista Apex's ZR-P™, a universal primer, for superior results.

**General Instructions (Bottle):**

1. After intra-oral try-in and determining the restoration is ready to deliver, thoroughly rinse with water to remove any contamination, and dry with oil-free air.
2. Dispense several drops of ZR-C™ into a well. Using an applicator brush, or micro applicator, liberally apply ZR-C™ to the entire intaglio surface of the restoration using a gentle brushing motion.
3. Allow to dwell for 20 seconds, then thoroughly rinse with water and dry with oil-free air.
4. Use Vista Apex's ZR-P™, a universal primer, for superior results.

**Target Patient Groups:** Patients undergoing restorative procedures using ceramic and metal substrates.

**Intended Users:** For professional dental use only.

**Storage:** Store between 2°C/36°F and 30°C/86°F. Keep container tightly closed. Keep only in the original container in a cool well-ventilated place. Keep container closed when not in use. Do not store with strong acids and oxidants. Do not use past the expiration date.

**Precautions:**

1. Only intended to be used by dental professionals.
2. Ensure adequate ventilation during use.
3. Wear protective eye wear and gloves when using Zr-C.
4. Avoid contact with eyes, skin, and soft tissue.
5. Application tips or brushes are single use only. Reuse of these items may result in cross-contamination between patients.
6. To avoid cross contamination, recap and wipe product with a suitable medical grade disinfectant between uses.
7. Wash hands and other exposed areas with mild soap and water before eating, drinking or smoking and when leaving work. Wash contaminated clothing before reuse. Do not eat, drink or smoke when using this product. Always wash hands after handling the product. Handle in accordance with good industrial hygiene and safety procedures.
8. Keep out of reach of children.

**Contraindications:**

Contraindicated for in vivo use; only intended for extraoral use.

---

**WARNING:**

Causes skin irritation. Causes serious eye irritation. May cause respiratory irritation. Avoid breathing dust/fume/gas/mist/vapors/spray. Wash skin thoroughly after handling. Use in a well-ventilated area. Wear protective gloves/eye protection/face protection. Disposal should be carried out in compliance with federal, state and local regulations. **FIRST AID: IF ON SKIN:** Wash with plenty of soap and water.

**IF INHALED:** Move to fresh air and keep at rest in a position comfortable for breathing.

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do so. Continue rinsing. If irritations persist, seek medical advice/attention.

**IF INGESTED:** Rinse mouth. Do not induce vomiting unless medical advice is given from a professional. Call a poison control center if feeling unwell and/or if irritation persists.

**Note:** Visit [vistaapex.com](http://vistaapex.com) for SDS.



**Rx ONLY**



**Inter-Med, Inc.**

2200 South St.

Racine, WI 53404, USA

Tel: +1-262-636-9755

Made in USA

91165-I-V-ENG (2)