

VISTA | APEX

CANAL DISINFECTION

AS EASY AS **1, 2, 3**

Save time by pairing Chlor-XTRA™
(ENHANCED 6% NaOCl) AND SmearOFF™
(2-IN-1: EDTA/CHX) the **ONLY** true 2-step
irrigation protocol.⁽¹⁾

- Dissolve tissue 2X⁽²⁾ faster and better
penetrate canal complexities⁽²⁾
- Disrupt biofilm and obtain a 3D clean
with EndoUltra® cordless ultrasonic
activator⁽³⁻⁶⁾




References: 1. Rizzo et al. 2017. PR61; AAE Annual Meeting Poster Presentation.
2. Pedulla et al. 2017. OR70; AAE Annual Meeting Oral Presentation.
3. Pedulla et al. 2017. Manuscript under review by IEL. 4. Int J Clin Dent (10): 45-54.5.
Hoshing et al. 2017. Manuscript under review by RDE.

For technical information, visit www.vistaapex.com
877-418-4782 Follow Vista Apex Products:



GOLD STANDARD IRRIGATION PROTOCOL

1

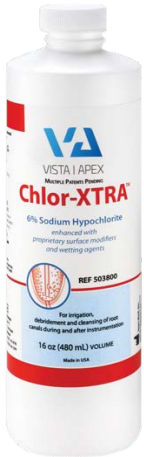

Chlor-XTRA™ (Enhanced 6% NaOCl)

Provides 2X faster tissue dissolution compared to standard NaOCl. Lowered surface tension enhances penetration into canal complexities.

EndoUltra®

Ultrasonic Activation improves irrigant penetration and canal disinfection through acoustic streaming and cavitation.



2



SmearOFF™ (EDTA/CHX mix)

Removes smear layer and kills bacteria in one easy step.

EndoUltra®

Ultrasonic Activation use for 30 sec-1 min per canal for a 3D clean.



Repeat Steps 1 and 2

3



Micro Suction

Dry using Elasti-Vac™ tip and Luer HVE adapter attached to low or high-speed suction.