



COMPOSITE & SETTING COMPATIBILITY



The chart below will list the ideal settings for most leading composite brands. Contact Vista Apex at (877) 418-4782 for additional assistance.

OPTIMAL Setting **Compatible** Setting **DO NOT USE**

COMPOSITE BRAND	SETTING 1	SETTING 2	SETTING 3	SETTING 4
Exquisite Restoration™ Nanofill Composite <i>Exquisite Restoration™ is a registered trademark of Inter-Med, Inc.</i>	Compatible	OPTIMAL	DO NOT USE	DO NOT USE
3M® <i>3M® is a registered trademark of 3M Company.</i>	Compatible	Compatible	OPTIMAL	DO NOT USE
DENTSPLY® <i>DENTSPLY® is a registered trademark of DENTSPLY International.</i>	Compatible	Compatible	OPTIMAL	DO NOT USE
Ivoclar® <i>Ivoclar® is a registered trademark of IVOCCLAR VIVADENT.</i>	Compatible	Compatible	OPTIMAL	Compatible
Kerr® <i>Kerr® is a registered trademark of Kerr Corporation.</i>	Compatible	OPTIMAL	DO NOT USE	DO NOT USE
VOCO® <i>VOCO® is a registered trademark of VOCO AMERICA, INC.</i>	Compatible	Compatible	Compatible	OPTIMAL
Coltene® <i>Coltene® is a registered trademark of COLTENE/WHALEDENT, INC.</i>	Compatible	OPTIMAL	DO NOT USE	DO NOT USE
Kuraray® <i>Kuraray® is a registered trademark of KURARAY CO., LTD.</i>	Compatible	Compatible	OPTIMAL	Compatible
GC® <i>GC® is a registered trademark of GC Corporation.</i>	Compatible	OPTIMAL	DO NOT USE	DO NOT USE
Shofu® <i>Shofu® is a registered trademark of SHOFU, INC.</i>	OPTIMAL	DO NOT USE	DO NOT USE	DO NOT USE
Heraeus Kulzer® <i>Heraeus Kulzer® is a registered trademark of Heraeus Kulzer GmbH.</i>	OPTIMAL	DO NOT USE	DO NOT USE	DO NOT USE
Tokuyama® <i>Tokuyama® is a registered trademark of Tokuyama Corporation.</i>	Compatible	OPTIMAL	DO NOT USE	DO NOT USE
Centrix® <i>Centrix® is a registered trademark of Centrix, Inc.</i>	Compatible	Compatible	OPTIMAL	DO NOT USE

CAUTION: Do not deviate from the recommended settings.