

SUPER CLEAN



ZR-C™

UNIVERSAL CLEANSER

Post Try-In Universal Cleanser

ZR-C™ vs. Market Leader*



Surface with Contaminants



Market Leader



ZR-C™
UNIVERSAL CLEANSER

* As tested via x-ray photoelectrospectrophotometry.

Removes 22% more contaminants after indirect try-ins for better prosthetic adhesion.



No shaking required



Can be used with ALL dental restorative materials



Superior removal of contaminants




Orange colorant provides an excellent contract to ensure complete coverage



Bottle Technique

1. Squeeze ZR-C™ from bottle directly into well, no shaking required.
2. Submerge desired brush into liquid and apply to restoration.
3. Allow to dwell for 20 seconds.
4. Rinse area completely after application.
5. Dry thoroughly.

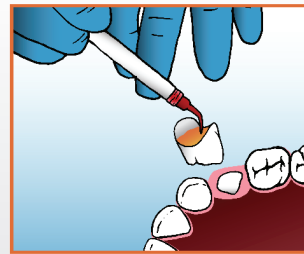

Syringe Technique

1. Use syringe to dispense ZR-C™ directly onto restoration.
2. Spread ZR-C™ to fully cover the restoration.
3. Allow to dwell for 20 seconds.
4. Rinse area completely after application.
5. Dry thoroughly.

ZR-C™ Universal Cleanser is an aqueous-based universal cleanser suitable for non-abrasive cleaning of the intaglio surface of prosthetic restorations after intraoral try-in.

#90799

ZR-C™ Bottle Kit
(1) 5g Bottle of ZR-C™

#90782

ZR-C™ Syringe Kit
(4) 1.2mL Syringes of ZR-C™
(25) Spira-Flo™ Brush Tips

Almost 20% Increase in Shear Bond Strength Over Competition

ZR-P™ Universal Primer is a MDP based dental primer for bonding to zirconia, alumina, and metal surfaces with resin materials. By combining three active monomers, ZR-P™ provides exceptional bond strength to the most challenging substrates.


- Provides high bond strengths to a variety of substrates
- Bond strengths that allow for the delivery of partial coverage indirects
- Enhances bond strengths of other adhesives and cements



SUPER STRONG

ZR-P™ UNIVERSAL PRIMER

ZR-P™ BOND STRENGTH VS. THE COMPETITION

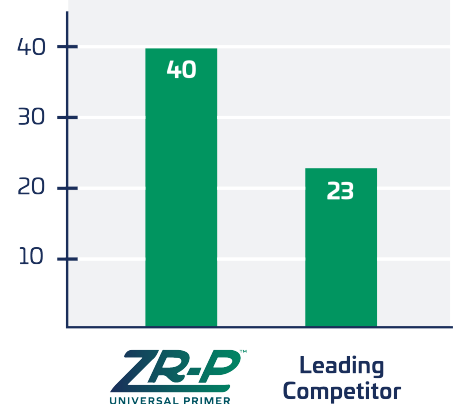

Provides high bond strengths to zirconia, alumina and metal restorations!

Use ZR-C & ZR-P Together to Make a Stronger Bond



#90577

ZR-P™ Bottle Kit
(1) 4mL Bottle of ZR-P™
(50) Applicators