

## PINKWAVE™ LED CURING LIGHT PROTOCOL

### Why PinkWave™?

PinkWave™ was distinctly designed to enhance the efficacy of light curing. PinkWave™ is the only curing light on the market that leverages *four different wavelengths* to reduce shrinkage.

**Outstanding Depth of Cure**  
8.5mm

**Reduced Composite Shrinkage**  
Decreased marginal staining, lower chance of microleakage and secondary caries

**Safe To Use**  
Very low heat emission

#### Step 1:



Make sure your PinkWave™ is fully charged (base will pulsate pink during charging, and stay solid pink when charging is complete). See IFU for complete instructions.



See PinkWave™ Accessories

Add your accessories (Barrier Sleeves, Light Shield and/or Anti-Roll Bracket).

#### Step 2:



Select your desired mode by cycling through presses of the Mode button (see below for recommendations for 3 Second Boost Mode, 20 Second Ramp Mode, 20 Second Standard Mode or Transilluminator Mode).



#### Step 3:



Cure your composite. PinkWave™ QuadWave™ Technology leverages four different wavelengths to ensure peak performance:

- Blue (CQ initiator)
- Red (Enhanced polymerization)
- Near-Infrared (NIR) (Enhanced polymerization)
- UV Light (UV-photo initiators)

#### Step 4:



When your procedure is complete, clean all contaminated parts of your PinkWave™ LED Curing Light. See IFU for complete guidelines.



As your PinkWave™ unit ages, much like any rechargeable battery, it loses some of its capacity over time.

Replacement batteries are available at a small fraction of the price of replacing an entire curing light.

Contact Customer Service for more information.

## PINKWAVE™ USER MODES

### 3 Sec Boost Mode



Boost Mode is a stronger but shorter curing light setting. This curing light mode is great for tack curing for your dental composite.

- 3s cure for every 2mm, followed by final (2) 3s cures
- 3s @ 1,720mW/cm<sup>2</sup>

### 20 Sec Ramp Mode



There is evidence that using Ramp Mode for composite restorations reduces shrinkage and minimizes post-operative sensitivity

- 20s cure for every 2mm, followed by final 20s cure
- Increases optical power from 5% to 100% in the first 5 seconds
- Remains at 100% (1,515 mW/cm<sup>2</sup>) for duration of cycle

### 20 Sec Standard Mode



Standard mode is the basic mode that keeps the same strength throughout the duration of the curing time. The device will beep once at 5s, twice at 10s, three times at 15s, and turn off at 20s.

- 20s @ 1,515 mW/cm<sup>2</sup>

### Transilluminator Mode



Use the Transilluminator Mode to identify cracks, interproximal decay and fractures, illuminate the canal opening and detect caries.

[vistaapex.com/pinkwave](http://vistaapex.com/pinkwave) | (877) 418-4782

Follow Us:



**pinkwave™**  
light curing evolved



## PINKWAVE™ ACCESSORIES


We are constantly evolving accessories to help you be even more efficient!



**Autoclavable Light Shield (5ct):**  
(SKU 90741) A small yet effective shield, placed over the curing light lens, to increase the protection of your eyes and reduce stray and reflecting light.



**Disposable Barrier Sleeves (100ct):**  
(SKU 90752-2) Disposable Barrier Sleeves are highly important for keeping your Pinkwave™ LED Curing Light sanitary. The barrier sleeve protects the curing light from the patient's tissues, and helps prevent composite or other materials from sticking to the lens.

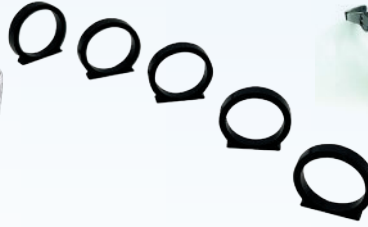

**Anti-Roll Brackets (5ct):**  
(SKU 90780-2) Prevent unfortunate accidents of your Pinkwave™ rolling off your countertop. One pack of 5 brackets comes with your Pinkwave™ Complete Kit purchase. It is also available for standalone purchase if you already own a Pinkwave™ LED Curing Light.



**Safety Glasses & Pinkwave™ Paddle**  
Three complimentary pairs of scratch-resistant safety glasses (SKU 90763) are included in the Pinkwave™ Complete Kit, but these are available for individual purchase as well. The Pinkwave™ Paddle (90786-2) is also available for dental professionals who prefer a larger protective area.

## CURING LIGHT LENSES

An exciting new accessory!  
Buy what you need, or buy the whole set!



**PinkWave™ Tack Tips (2/pk)**  
(SKU 90796-2) Gives you a 2.5mm diameter curing area to aid in position and tack curing during your dental procedures.



**Tight Contact Curing Ball Tips (2/pk)**  
(SKU 90795-2) Helps you hold the dental matrix against contact while curing composite for Class II restorations.



**Interproximal Diagnostic Lenses (2/pk)**  
(SKU 90794-2) Gives you a focused, 1mm white light to help view interproximal caries when using the Transilluminator Mode.



**Complete Set (6/pk)**  
(SKU 90793-2) Includes two of each PinkWave™ Curing Light Lens.

All lenses are reusable with disinfection using intermediate-level disinfectant after each dental procedure.

## TESTIMONIALS

See what your peers have to say about the Pinkwave™!  
Visit our landing page to watch these videos or to learn more!


"Pinkwave™ has high output power, so I know that it cures all of my bulk fill, and it's got a broad spectrum of cure, so I know it cures all the different photo initiator systems out there."



**Dr. Josh Austin**  
Vista Apex KOL

"I can rely on Pinkwave™ in my practice, and so can you!"

"Pinkwave™ is one of the most powerful tools that all dentists need in terms of adhesive-based dentistry. Increase efficiency and productivity in your practice today."

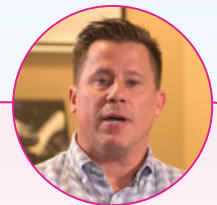

**Dr. Troy Schmedding**  
Vista Apex KOL

**VISTA APEX**  
vistaapex.com/pinkwave | (877) 418-4782

Follow Us:



**pinkwave™**  
light curing evolved

